




Fitting Protocol

Basic Sling
Models 25045, 25050, 25070

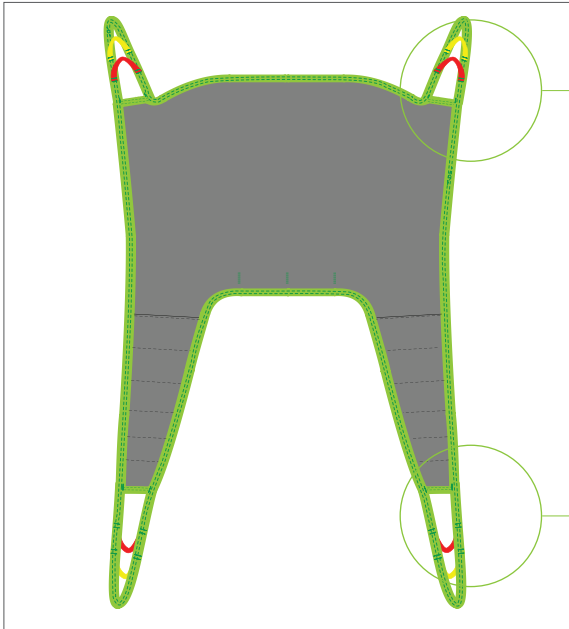


- Green Strap
- Yellow Strap
- Red Strap

Extension Loops

- 21180-10 (10cm/3.9")
- 21180-15 (15cm/5.9")
- 21180-20 (20cm/7.8")
- 21180-25 (25cm/9.8")
- 21180-35 (35cm/13.7")

Choose the loop combination that will give the most comfortable and safe suspension angle, keeping in mind the patient's ability to support themselves.

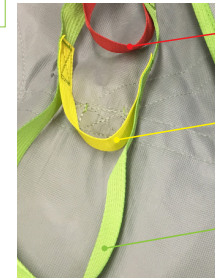

- Red Strap**
Upright Position
Least hip angle
- Yellow Strap**
Semi-Recumbent
Mid-hip angle
- Green Strap**
Increased Recline
Less hip flex and greatest angle

- Green Strap
- Yellow Strap
- Red Strap

Extension Loops

- 21180-10 (10cm/3.9")
- 21180-15 (15cm/5.9")
- 21180-20 (20cm/7.8")
- 21180-25 (25cm/9.8")
- 21180-35 (35cm/13.7")

Match colours on both sides of the patient for a 90 degree angle in the hips.

Maximum User Weight: 250kg/551lb

Model 25045 - Polyester Net

Model 25070 - Polyester

Model 25050 - Trevira

Transfer Situation:

- To/From a Bed
- To/From a Chair
- To/From the Floor
- Showering (25045 only)

Available sizes:

Indicate size required

- JS
- JM
- S
- M
- L
- XL

Leg Placement:

- Divided relaxed position
- Open position
- Divided position
- Closed position

Suspension:

- 2-Point
 - 4-Point
 - 3-Point
- Not available for this sling**

Accessories required:

- 21146 Leg Padding - Small
- 21146 Leg Padding - Medium
- 21146 Leg Padding - Large
- 25102 Reinforced Leg Support - One Size
- 50866 Upholstered Cover for Hanger Bar
Do not use in wet environments.

Patient Information:

Name

Facility Room

Lift Used Hanger Bar Used

HUMAN CARE SWEDEN (HQ)

Årstaängsvägen 21B
117 43 Stockholm
Phone: +46 8 665 35 00
Fax: +46 8 665 35 10
info.se@humancaregroup.com

HUMAN CARE AUSTRALIA

Unit 4, 49 Bertie Street
Port Melbourne, VIC 3207
Phone: +61 1300 132 736
Fax: + 61 (03) 9645 4954
info.au@humancaregroup.com

HUMAN CARE CANADA

10-155 Colonnade Road
Ottawa, ON K2E 7K1
Phone: +613.723.6734
Fax: +613.723.1058
info.ca@humancaregroup.com

HUMAN CARE NEDERLAND

Elsperweg 124
8076 PA Vierhouten
Phone: +31 577 412 171
Fax: +31 577 412 170
info.nl@humancaregroup.com

HUMAN CARE UNITED STATES

8006 Cameron Road, Suite K
Austin, TX 78754
Phone: +512.476.7199
Fax: +512.476.7190
info.us@humancaregroup.com

The Basic Sling is a comfortable sling without reinforcements and handles. The sling adapts to the patient and offers good support for the entire back. The Basic Sling can be used in most lifting situations, fits most patients and is a good choice if it is not established who is going to be using the sling. The Basic Sling is offered in Polyester Net and Trevira fabric that allow moisture and heat to transfer efficiently from the patient. This allows the sling to be left behind the patient for a short period of time after the transfer is completed. The Polyester material is durable, easy to apply and maintain.

Additional Instructions

To staff/caregiver:

To patient:

Fitting Protocol Completed by:

Name _____

Date _____ Position/Title _____



Important! This fitting protocol does not replace the user manual. Read all instructions before use!

Additional Documentation

Can be found on our website at www.humancaregroup.com/product-category/slings



Quick Reference Guide



User Manual



Sling Care Instructions



Sling Inspection Form

