
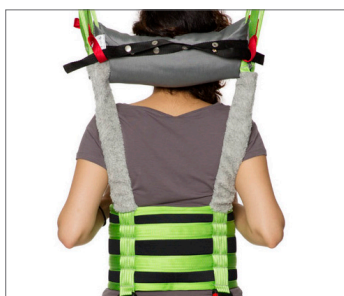


multi slings

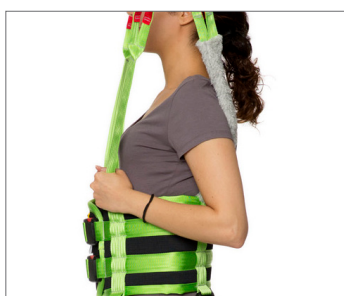

The Multi Sling is the perfect sling for gait training and rehabilitation exercises. It is suitable for patients with some degree of body stability and can be used for dressing and undressing and visits to the toilet. It is made of durable Polyester and comes with a full assortment of accessories for additional support and versatility.

Multi Use

The Multi Sling is designed to act as support while doing standing, walking and balancing exercises, however, it can also be used for dressing and undressing and visits to the toilet. It consists of different parts - a chest belt, groin support and shoulder support. The Multi Sling can be used with either 2-point, 3-point or 4-point suspension and the user of this sling must have partial use of their legs.

Full Range of Accessories

The Multi Sling offers a full range of optional accessories that increase its functionality. The chest belt can be used as a support during walking and balancing exercises and, when combined with the hip belt, it allows

patients with functional impairments (such as stroke) to use the product. For extra comfort the neck cushion can be used and for added security, the groin support or a waist belt.

Easy Care and Easy Fit

Our slings are manufactured to the highest quality standards. They feature reinforced straps and high strength nylon webbing around the sling to make them strong enough to resist even the most active use. Each sling label is colour coded to ensure quick and easy selection of the correct sling and the straps are colour coded for easy determination of required positioning.

HUMAN CARE SWEDEN (HQ)

Årstaängsvägen 21B
117 43 Stockholm
Phone: +46 8 665 35 00
Fax: +46 8 665 35 10
info.se@humancaregroup.com

HUMAN CARE AUSTRALIA

Unit 4, 49 Bertie Street
Port Melbourne, VIC 3207
Phone: +61 1300 132 736
Fax: +61 (03) 9645 4954
info.au@humancaregroup.com

HUMAN CARE CANADA

10-155 Colonnade Road
Ottawa, ON K2E 7K1
Phone: +613.723.6734
Fax: +613.723.1058
info.ca@humancaregroup.com

HUMAN CARE NEDERLAND

Elspeterweg 124
8076 PA Vierhouten
Phone: +31 577 412 171
Fax: +31 577 412 170
info.nl@humancaregroup.com

HUMAN CARE UNITED STATES


8006 Cameron Road, Suite K
Austin, TX 78754
Phone: +512.476.7199
Fax: +512.476.7190
info.us@humancaregroup.com

Accessories



Extension Loops

Used to extend lift straps in order to alter lifting position or to hold leg parts together. Assorted lengths. Part no. 21180.*

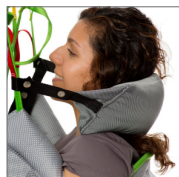

Neck Cushion

For added comfort and support for the neck. Part no. 25150



Waist or Hip Belt

Waist belt with Velcro (Part no. 25145**) or with Safety Lock (Part no. 25146***). Hip belt Part no. 25107.



Upholstered Cover

Protects user from hanger bar impact. Not for use in a wet environment. Part no. 50866

Lifting Situations



Use to/from a bed

Suitable for transfers and repositioning to and from a bed. Can be left behind patient for short period of time.




Use to/from a chair

Suitable for transfers and repositioning to and from a chair. Can be left behind patient for short period of time.



Use for gait training/ mobilization


Suitable for gait training and rehabilitation exercises.

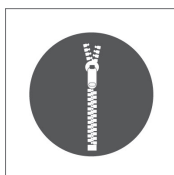

Use for dressing/ undressing


Suitable for dressing/undressing applications.

Care and Features

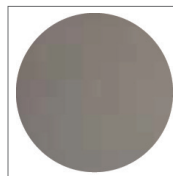


Polyester

Durable, low-friction, easy to clean, apply and remove.

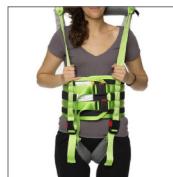


Components

Part no.
25106 Chest Belt,
25108 Groin Support,
25109 Shoulder Support

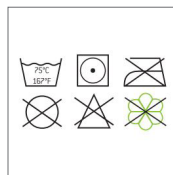

Care


Care instructions located on sling label. Use a hospital grade laundry bag while laundering.

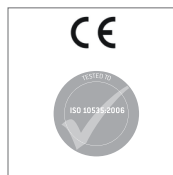

Tested to all relevant standards, including ISO 10535, hoists for the transfer of disabled persons.

Technical Specifications

Part Number	Size	Colour	Weight Range	Waist Range	A	B	C
Polyester			[Recommended]	[Min-Max]			
25105-S	Small		30kg -60kg/ 66.1 lb - 132.3 lb	70cm - 86cm / 27.6" - 33.9"	86cm / 33.9"	36cm / 14.2"	20cm / 7.9"
25105-M	Medium		55kg -90kg/ 121.3 lb - 198.4 lb	86cm - 104cm / 33.9" - 40.9"	104cm / 40.9"	38cm / 15"	20cm / 7.9"
25105-L	Large		85kg - 120kg/ 187.4 lb - 264.6 lb	104cm - 122cm / 40.9" - 48"	122cm / 48"	40cm / 15.8"	20cm / 7.9"
25105-XL	Extra Large		120kg - 150kg/ 264.6 lb - 330.7 lb	122cm - 140cm / 48" - 55.1"	140cm / 55.1"	42cm / 16.5"	20cm / 7.9"
Maximum User Weight		150kg/330.7lb					

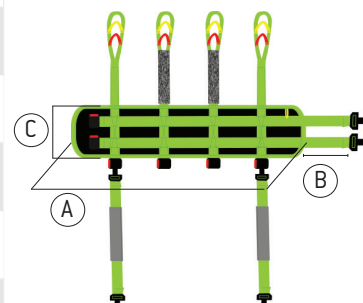

* Available in assorted lengths: 10cm/3.9" [21180-10], 15cm/5.9" [21180-15], 20cm/7.8" [21180-20], 25cm/9.8" [21180-25], 35cm/13.7" [21180-35]

** Available Sizes: 60cm/23.6"/[25145-60], 70cm/27.6" [25145-70], 80cm/31.5" [25145-80], 90cm/35.4" [25145-90], 110cm/43.3" [25145-110], 120cm/47.3" [25145-120]

*** Available Sizes: 60cm/23.6"/[25146-60], 70cm/27.6" [25146-70], 80cm/31.5" [25146-80], 90cm/35.4" [25146-90], 100cm/39.4" [25146-100], 110cm/43.3" [25146-110], 120cm/47.3" [25146-120]

This product is manufactured in accordance with all relevant ISO standards.



Learn More About This Product



Follow Us on Social Media



facebook.com/humancareworld



instagram.com/humancaregroup



linkedin.com/company/human-care-ab



pinterest.com/humancaregroup



youtube.com/channel/UC7fjInh3zTjRxOP1IDf2Da0