



Our Limb Sling is designed to comfortably and conveniently assist in lifting, supporting and positioning a client's body part like legs, arms, shoulders, pannus or hips. This allows better access for treatment, performing surgery, leg/venous wrapping, foot care, wound care treatment, hygiene and a variety of other care-related tasks.

#### Comfortable Design

The easy care material is gentle against the skin and can function equally well on both the upper and lower extremities. Its padded design is comfortable to use and its flexibility allows it to contour to the body shape and angles needed for wound care. A non-absorbent, water resilient filling and a CFC, HCFC and UV resistant material make it safe to use in many environments.

#### Multi-Functional

With a cooperative client who has good trunk control, endurance and cognition, this sling can help with turning, for elevating a shoulder for access to the head, or for elevating hips for a slight pelvic bridge position. It is recommended that the limb sling is used with an overhead lift or floor lift to promote

caregiver safety and to avoid patient injury and/or skin friction. Alternate applications include, support for the pannus (manual lifting), for rehabilitation exercises, support for patient movement while standing up from a chair or bed (by placing around their shoulders), or to lift a limb while getting out of bed. The limited amount of material allows easy application by a single caregiver.

#### Caregiver Safety

Caregiver safety is an important concern in facilities. Using the Limb Sling reduces the caregiver's physical load by lessening their movements of bending and leaning forward as it acts as an extension for their arms while transferring the patient's body part to the correct location and height.

#### HUMAN CARE SWEDEN (HQ)

Årstaängsvägen 21B  
117 43 Stockholm  
Phone:- +46 8 665 35 00  
Fax:- +46 8 665 35 10  
info.se@humancaregroup.com

#### HUMAN CARE AUSTRALIA

Unit 4, 49 Bertie Street  
Port Melbourne, VIC 3207  
Phone : +61 1300 132 736  
Fax : + 61 (03) 9645 4954  
info.au@humancaregroup.com

#### HUMAN CARE CANADA

10-155 Colonnade Road  
Ottawa, ON K2E 7K1  
Phone: +613.723.6734  
Fax: +613.723.1058  
info.ca@humancaregroup.com

#### HUMAN CARE NEDERLAND

Elspeterweg 124  
8076 PA Vierhouten  
Phone: +31 577 412 171  
Fax: +31 577 412 170  
info.nl@humancaregroup.com

#### HUMAN CARE UNITED STATES

8006 Cameron Road, Suite K  
Austin, TX 78754  
Phone: +512.476.7199  
Fax: +512.476.7190  
info.us@humancaregroup.com

## Accessories



### Ceiling Lifts

Compatible for use with our Altair, Roomer S or HeliQ ceiling lifts



### Floor Lifts

Compatible for use with most floor lifts, including our FL180 and Lexa Pro.

## Lifting Situations



### Use for support

Suitable as support for one or both legs or one arm for such activities as wound treatment, dressing and hygiene.



### Alternate support


Suitable as support for shoulders, pannus and hips for care related functions.



### Use for mobilization


Suitable for mobilization exercises while in a bed.

## Care and Features

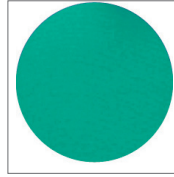

### Polyester Tricot

CFC and HCFC free. Landau underpad, Polyfilm foam filling. Nylon binding.

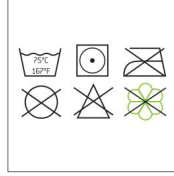

### Care

Care instructions located on sling label. Use a hospital grade laundry bag while laundering.




Tested to all relevant standards, including ISO 10535, hoists for the transfer of disabled persons.



UV resistant material.

## Technical Specifications
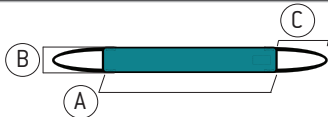
| Part Number | Size     | A             | B           | C                              |  |
|-------------|----------|---------------|-------------|--------------------------------|--|
| NSB-010     | One Size | 123cm / 48.4" | 17cm / 3.7" | 45.5cm / 18"<br>(pulled tight) |  |

This product is manufactured in accordance with all relevant ISO standards.



Learn More About This Product



### Follow Us on Social Media



facebook.com/humancareusa



instagram.com/humancaregroup



linkedin.com/company/humancareusa



pinterest.com/humancaregroup



youtube.com/channel/UC7fyiNh3zTjRxOP1IDr2Da0