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## Table of Contents

### **1 General Information**

- 1.1 Product versions
- 1.2 Accessories
- 1.3 Scope of delivery
- 1.4 Storage, transport
- 1.5 Service life / lifecycle

### **2 Safety information**

- 2.1 Symbols used
- 2.2 Function check

### **3 Using the product**

- 3.1 Indication and further use
- 3.2 Contraindications
- 3.3 Application risks
- 3.4 Areas of application
- 3.5 Application illustration
- 3.6 Application
- 3.7 Approved accessories, combination with other products

### **4 Product marking / CE**

### **5 Cleaning, care and disinfection**

### **6 Technical Data**

### **7 Warranty**

### **8 Disposal information**

# 1 General Information

Thank you for choosing a Petermann product.

## **Please read the user manual carefully before using the product for the first time!**

The current understanding in modern occupational health and safety is that people in nursing professions experience significant work-related physical strain. Back injuries are common. There are many situations in which a patient has to be transferred from a bed, a wheeled stretcher, ambulance stretcher, an X-Ray table or a wheelchair to a different stretcher or seat. The strain on the user and the patient / care-dependent can be reduced significantly through the use of sliding mats.

The use of transfer aids should be conditional upon the user having learned to use the equipment properly. The user has to decide for themselves what equipment is suitable to the patient's situation and the intended sequence of movements.

The inner surface of all Turning Sheet with hand grips is made from particularly low friction fabric, which allows for easy movement. The outer surfaces consist of a breathable cotton material, which can remain under the patient. The Turning Sheets with hand grips have soft padding between the surfaces so that the patient can lie or sit comfortably.

The Turning Sheet with hand grips is part of a product line which is used specifically to aid in the transfer of patients in home care and other health care facilities, as well as in hospitals.

The product may only be used for the specified purpose and under the specified conditions.

The Turning Sheet with hand grips is a safe transfer aid, which allows for a transfer which is pleasant and comfortable for the patient as well as being simple and back-friendly for the users.

The information in this user manual is for the patient who is being transferred and moved, as well as the user performing the transfer / position change.

Contact us if you require additional information or assistance using the Petermann product.

## 1.1 Product versions

**Item no.: PM-2098.2** Turning Sheet with hand grips

## 1.2 Accessories

**Item-no.: PM-2091** Fastening loop set

## 1.3 Scope of delivery

Turning Sheet with 12 hand grips (6 on each side)

2 Fastening loop set

User manual

## 1.4 Storage, transport

Store/transport at -5 C – 40 C  
Other: protect from sunlight

## 1.5 Service life / lifecycle

If the product is used in accordance with the safety instructions, then the expected product service life is **five years**. However, the effective service life may vary depending on the frequency and intensity of use and the number of washing and disinfection cycles.

Perform regular function checks on the product and be sure to observe the safety information (see item 2).

## 2 Safety information

**WARNING indicates a potential danger which could lead to personal injury or material damage.**

- It is the responsibility of the organisation / the user to determine, on the basis of the circumstances (weight / health condition of the patient) and the capabilities of the user, whether several users may be required for the transfer and/or whether additional safety measures are required.
- Defective products may no longer be used.
- The smooth surfaces significantly reduce resistance due to friction during repositioning. Check the transfer surfaces carefully to ensure they are fixed in place securely (e.g. wheelchair, bed).
- Ensure that the transfers are performed by the users in a back-friendly manner.
- Weight limits must be observed (200 kg).
- Transfer products are subject to a natural ageing process. Inspect the equipment regularly to ensure proper functionality.
- The use of non-standard cleaning agents or bleaching agents, alcohols, fabric softeners or other chemical laundry additives may lead to permanent deterioration of the sliding mat properties.

## 2.1 Symbols used

	90°C laundry		max. load capacity see item 6
	Do not bleach		Manufacturer
	Dry at low temperatures		Batch no. / lot no.
	Do not iron		Identification code
	Do not dry clean		Conformity mark
	Warning		Item number
	Observe the user manual		Medical device label

## 2.2 Function check

Transfer products are also subject to a natural ageing process. Inspect the equipment regularly and carefully, in particular before each use. Ensure that the product is in sound condition and that the material and the seams are not damaged. Check the product functionality by applying pressure to the hand grips and ensure that the material withstands the load. The smooth surface of the special sliding material allows for easy sliding and moving. Check these properties - ideally in comparison to a new product.

➤ **Defective products may no longer be used!**

## 3 Using the product

### 3.1 Indication and further use

Due to the wide range of possible applications, the indications cannot be tied to a specific clinical picture, since Turning Sheets with hand grips are suitable for all applications involving people who require assistance. Thanks to the reduced frictional resistance, the capabilities of the patient can be put to use when Turning Sheets with hand grips are used. The effort required on the part of the user is reduced. The product can be used in support of the following diagnoses.

- For poor skin condition
- For large area wounds
- With decubitus
- For patients with an increased risk of bleeding due to skin contact
- For micro position changes
- For individually felt pain

- To the transfer of overweight patients
- As an option for recumbent transfer from bed to an adaptive wheelchair
- For mobilisation and positioning of a patient in bed according to indications
- For patients with elevated or reduced muscle tone
- For patients unable to move on their own
- Can be used with minimal independent patient mobility (e.g. ALS or MS)
- To reduce shear and friction forces
- Application in palliative settings
- A combination of pressure relieving mattresses, pain treatment systems or alternating pressure mattresses is possible in specific configurations after testing.

**This product may be reused.**

Before reusing the product, be sure to perform a thorough function check (see item 2.2) as well as cleaning or disinfection of the product. (see item 5)

### 3.2 Contraindications

Contraindications cannot be tied to a specific clinical picture. Under consideration of the safety information (see item 2), the product may not be used if the patient has any of the following conditions:

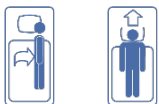
- Independent/active patients

### 3.3 Application risks

- Patients are miscalculated → risk of falling
- The product is embedded incorrectly → risk of falling

### 3.4 Areas of application

The product can be used in home care and other health care facilities as well as hospitals.

### 3.5 Application illustration

## 3.6 Application

Using the Turning Sheet with hand grips doesn't involve any prescribed care techniques or methods, it is based on the individual circumstances. Moving and lateral repositioning of patients is made much easier. The Turning Sheet with hand grips can be secured at the head of the bed or on the sides using the included fastening straps. Transfers and positioning of the patient are performed the same way as for the aforementioned applications. The Turning Sheet with hand grips can remain in the bed permanently.

The Turning Sheet with hand grips is very versatile. Depending on the type of restriction, weight and sensitivity to pain we recommend using it as a complete set with a Fitted Sheet, Bed Wedges and Non Slip Mat.

The sliding surface of the fitted sheet is finished with extremely low-friction sliding material. The fact that the sliding surfaces of the top sheet and the Turning Sheet with hand grips lie against one another maximises the reduction of friction forces, producing the same effects as a sliding mat. An anti-slip safety strip is sewn into the sliding surface of the fitted sheet on both sides. The safety strips prevent the top sheet from sliding off in an uncontrolled manner under load and facilitates all transfers in and out of the bed.

Placing the Non slip mat between the Turning Sheet with hand grips and the Fitted Sheet reduces slippage in this area.

The Bed Wedges were developed for active and corrective positioning of patients. Their versatility allows for goal-oriented and purposeful positioning in accordance with the planned care measures. (also refer to the Bed Wedges user manual)

### 3.6.1 Applications

When using the Turning Sheet with hand grips in combination with the Turning Sheet Fitted Sheet, Anti-Slip Mat and a Bed Wedges, the following parts are used:

- Turning Sheet with hand grips, 3 fixation loops at the head end and 12 hidden hand grips
- Fitted sheet with sliding surface and anti-slip safety strip
- Non-Slip mat to aid with supporting and positioning
- Bed Wedges for positioning the patient

### 3.6.2 Preparation

- Fit the Fitted Sheet onto the mattress like a fitted bed sheet.
- Tighten it on the sides using the pull cords.
- Place the Turning Sheet with hand grips directly onto the fitted sheet.
- Place the bed sheet and incontinence sheet on the Turning Sheet with hand grips as needed.

### 3.6.3 Transfer

- With the Turning Sheet with hand grips, the patient can be pulled in all directions as well as being pushed.
- The movements can be adjusted according to the respective clinical picture. They should always be performed gently and in small steps.
- To pull, grasp the pad beside the patient's centre of gravity; Then the user can pull the patient towards themselves by shifting their centre of gravity.
- To push, place the hands flat under the patient's centre of gravity; The patient can be pushed by shifting one's weight and applying pressure.
- Please ensure that the pad is pushed, and not the patient!
- To pull the patient up, pulling the Turning Sheet with hand grips up towards the head of the bed is recommended. If this isn't possible, then the Turning Sheet with hand grips can be pushed up diagonally from the side.
- The Turning Sheet with hand grips can be secured on the head end using the supplied straps.
- Patients with some mobility can also actively support their movements with the aid of the pad.

- Furthermore, movement approaches are physiologically supported by the Turning Sheet with hand grips.
- 

Visit our website at [www.pm-med.de](http://www.pm-med.de) or watch our comprehensive product videos on our YouTube channel.

### 3.7 Approved accessories, combination with other products

Turning Sheets with hand grips are used in combination with the following products:

- PM-7010/7011 Non-Slip Mats
- PM-1584 Fitted sheet
- PM-8012/8015 Bed Wedges

Petermann transfer aids are part of a comprehensive aid product range, which are effective and back-friendly for the user and comfortable and supportive for the patients. Additional Petermann products may be used according to the situation and clinical picture. Ask our experts. We are happy to help you!

## 4 Product marking / CE

The Petermann Turning Sheet with hand grips is a Class I Medical Device and meets the EU-Regulation 2017/745 of the European Parliament and Council dated 05.04.2017 for medical devices, also called MDR.

Petermann GmbH is certified in accordance with DIN EN ISO 13485 by TÜV Süd, Medical Devices - Quality management systems applicable to the design and development, production and distribution of aids for people of limited mobility.

## 5 Cleaning, care and disinfection

The product doesn't require any special maintenance. It can be machine washed at 90°C and dried at low temperatures (45° - 50°C). We recommend washing with mild detergents. The use of cleaning agents with bleaching ingredients or other chemical laundry additives, also as fabric softeners may lead to permanent deterioration of the product properties.

When necessary, we recommend wipe disinfection with the product Trionic® (DGHM/VAH approval, further information upon request). As alternatives to our proven disinfectant Trionic®, we can also recommend the following disinfectants for wipe disinfection:

- Mikrobac® Tissues\*
- Mikrobac® forte\*
- BacilloI® 30 Foam
- BacilloI® 30 Tissues

When using products marked with an \*, wipe down again using a water-soaked cloth after disinfection has been completed, in order to remove residues and prevent build-up. For wipe disinfection, please be sure to always follow the instructions provided by the manufacturer.

Always follow the care instructions on the label of every product as well as the information in the respective user manual.

## 6 Technical Data

Item no.	Material	Colour	Load-bearing capacity
PM-2098.2	Cotton, natural thicking, latex free	cream	200 kg

Item no.	Description	Dimensions in mm	Weight	UDI
PM-2098.2	Turning Sheet with hand grips	2000 x 1000 x 5	1650 g	(01)4250355320988(10)x

The manufacturing of our products is subject to highest quality standards. Due to usual manufacturing variations published dimensions and further information concerning the condition of the product are defined as target production specifications. Deviations which might occur are monitored by professional quality management procedures and do not represent a reduction in performance of the respective product.

## 7 Warranty

- The company Petermann warrants that the product is free from defects in material and workmanship.
- The warranty period is 3 years. **It does not cover defects attributable to abrasion, normal wear, improper use or improper cleaning.**
- The company Petermann is not liable for accidental or consequential injury or material damage.

## 8 Disposal information

There are no specific disposal regulations for Turning Sheets with hand grips, they can be disposed of in accordance with applicable national and local regulations.

**Petermann GmbH**  
**Waldweg 3**  
**D-91601 Dombühl**  
**Phone: +49 (0) 9868-9339-0**  
**Email: petermann@pm-med.de**

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