

# FL180



Flexible  
Easy to Use  
Versatile



Our FL180 floor lift is a stable and flexible floor lift that can be used in all types of lifting situations. It allows for lifting from high surfaces, such as a chair, bed, wheelchair or commode, in addition to lifting from the floor. The FL180 is the perfect solution as a complement to our overhead lift for several lifting situations.

### Flexible

The FL180 easily lifts from both low and high surfaces to move a patient from a bed, seated position or even the floor. It works seamlessly with most of our loop slings, allowing it to be used in many different lifting situations. The low floor height ensures it easily fits under most furniture and its portability allows it to easily be moved between different rooms or departments in facilities. It is an optimal solution when overhead lift installations are not possible or planned or for use in corridors. It also offers a versatile temporary solution when a lift is required only for a short period of time or when waiting for a permanent solution.

### Easy to Use

The FL180 features LINAK actuators and easy

to maneuver wheels, two of which feature a braking system for security when loading or unloading a patient. Its base can be adjusted electronically and it features manual or electric emergency lowering. The display panel clearly indicates battery levels and service requirements.

### Quality

Our products are designed and manufactured to meet all the requirements of relevant ISO, UL, IEC and EN standards. At every stage of the development process, every single product is given extensive attention in design, development and manufacturing. This means that a user of this product feels confident with its reliability and the caregiver feels confident with its safety.

#### HUMAN CARE SWEDEN (HQ)

Årstaängsvägen 21B  
117 43 Stockholm  
Phone:- +46 8 665 35 00  
Fax:- +46 8 665 35 10  
info.se@humancaregroup.com

#### HUMAN CARE CANADA

10-155 Colonnade Road  
Ottawa, ON K2E 7K1  
Phone: +613.723.6734  
Fax: +613.723.1058  
info.ca@humancaregroup.com

#### HUMAN CARE NEDERLAND

Elspeterweg 124  
8076 PA Vierhouten  
Phone: +31 577 412 171  
Fax: +31 577 412 170  
info.nl@humancaregroup.com

#### HUMAN CARE UNITED STATES

8006 Cameron Road, Suite K  
Austin, TX 78754  
Phone: +512.476.7199  
Fax: +512.476.7190  
info.us@humancaregroup.com

## Accessories



### Hanger Bars

Comes complete with 50cm/19.7" hanger bar. Optional small (70977) and medium (70978).



### Upholstered Cover

Protects user from injury. Not for use in wet environments. Does not work with small (70979). Part no. 50866



### Slings

Assorted sizes and models of Human Care slings can be used with this product.



### Hand Control

Included with floor lift. Lift arm up/down, leg mast open/closed functions. Replacement Part no. 70974

## Lifting Situations



### Use to/from a bed

Use to raise/lower a patient from/to a bed.



### Use to/from a chair

Use to raise/lower a patient from/to a chair.



### Use to/from the floor


Use to raise/lower a patient from the floor.



### Use to/from the toilet

Use to raise/lower a patient from/to a toilet.

## Features

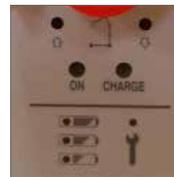

### LED indicator

Indicates low battery level and service requirement.



### Emergency Stop

Cuts power to lift for emergency situations. Reset by turning counter clockwise.



### Emergency Lowering

Manual or electric.



Tested to all relevant standards, including ISO 10535, hoists for the transfer of disabled persons.

## Technical Specifications

FL180 Floor Lift			
Part No.	70967-[Country Code]	Lifting Interval	132cm / 52"
Width	63cm / 24.8"	Turning Radius	120cm / 47.2"
Length	114cm / 44.9"	Wheels	7.5cm / 3" (twin wheels), rear with foot-operated brake
Height	Arm Lowered: 145cm / 57.1" Arm Raised <sup>1</sup> : 199cm / 78.3"	Battery	Lead-acid, 2.9Ah 24V Replacement Part no. 70975
Height from Floor	11.5cm / 4.5"	Hanger Bars	Small 37cm/14.6" Part no. 70977
Adjustable Leg (Min - Max)	Inside: 52cm - 99cm / 20.5" - 39" Outside: 63cm - 111cm / 24.8" - 43.7"		Medium 47cm/18.5" Part no. 70978
Weight	32kg / 70.5lb		Large 50cm/19.7" Part no. 70979
Weight Capacity	180kg / 397lb	Power Cable	Replacement Part no. 70980 (EU), 70981 (UK), 70982 (US), 70983 (AU)
IP Rating	IPX4		

<sup>1</sup> Measurement based on arm mast at highest setting