



Rebel RA&TW

 To secure that the walking-aid will function safely for the user, once per year review of the walking-aid should be done. Therefore is the following interventions recommended.

Cleaning

When cleaning the walker it is recommended machine washing, 60 degrees for 10 minutes. When using a pressure washer, do not wash directly to the bearings. In addition to machine washing, regular cleaning with a mild cleanser is recommended. **Note!** Do NOT use thinner, petrol or other solvents.

Post treatment

After washing, all connections, tight spaces and drainage holes should be blown dry and sprayed with thin oil. If degreaser is used, should the moving parts be lubricated.

The Paint

Control damages to the paint. To prevent corrosion, improve scratches. Touch-up paint can be ordered from Human Care HC AB.

Pipe frame

Control weldings and screw attaches. If the corrosion is extensive shall the rollator be replaced.



Handle

Must be fixed. Make sure the handle pipes can be pushed down to the chassis.

Brakes

Control that both brakes and parkingbrakes are working. Wires should be intact and strapped to the chassis.

Wing nut

Control function.

Reflectors

Make sure the reflectors are still there and visible.

CE-marking

Make sure that the CE-marking is still there and readable.

Wheels

Make sure the wheels are not worn-out and are spinning easily without noise.

Lubrication

Gatebushings and hinges should be working without any noise. Lubricate when needed.

Front forks

Make sure that attachments and bearings does not fit loosely and are spinning easily.

Folding mechanism

Should be intact and the locking should function.

Seat

Make sure that the seat is kept in place and the nuts are tightened.

Stability

Control that all wheels are steadily on the ground and that there are no chassis distortion.


Test driving

Make a final control by test driving.



RECONDITIONING MANUAL

No	Designation	Quantity	Art. no	Other information
	TW			
18	Walking table cushion	1	20-053	
19	Cushion screw	4	00818	
20	Bracket compl right	1	100-001H	
21	Bracket compl left	1	100-001V	
22	Teleskopisk tube	2	20-010K	Rebel 67 RA/TW Low
22	Teleskopisk tube	2	20-010L	Rebel 67 RA/TW Normal
22	Teleskopisk tube	2	20-011K	Rebel 72 RA/TW Low
22	Teleskopisk tube	2	20-011L	Rebel 72 RA/TW Normal
	Wing nut complete	2	50937	Lubricate the wing nut when needed.
23	Wing nut	2		
24	Screw	2		
25	Washer	2		
26	Wire housing	2	00011	Rebel 67/72 RA/TW Low
26	Wire housing	2	00012	Rebel 67/72 RA/TW Normal
26	Wire	2	00005	Rebel 67/72 RA/TW Low/Normal
	Wire lock	2	10-017	
	Wire end	2	00205	
	Seat kit	1	89020	
27	Seat	1		
28	Gate lock	1		
29	Fold loop	1		
30	Screw M6x20	2		
31	Screw to gate lock	4		
	Seat complete	1	89070	
27	Seat	1		
30	Screw M6x20	2		
	Tube protection	1	50949	
29	Fold loop	1	10-009	
	Gate lock kit	1	10-018	
28	Gate lock	1		
31	Screw to gate lock	4		
29	Fold loop	1		
32	Gate upper complete	1	20-047MB	Rebel RA/TW 67
32	Gate upper complete	1	20-047B	Rebel RA/TW 72
32	Gate upper complete -seat	1	20-037MB	Rebel RA/TW 67
32	Gate upper complete -seat	1	20-037B	Rebel RA/TW 72
	Reflex	2	00028	
	Touch-up paint		F2	

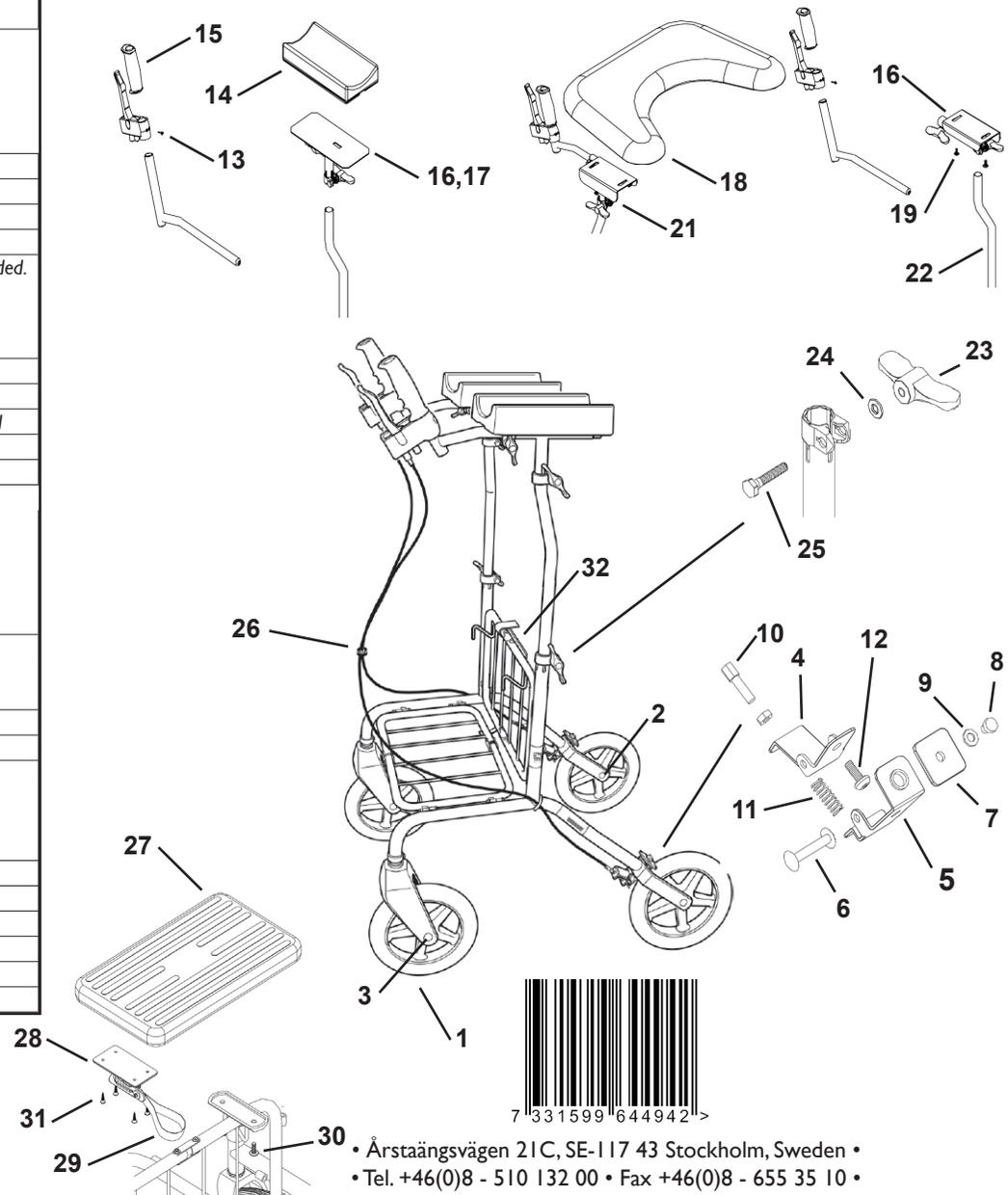

• Årstaängsvägen 21C, SE-117 43 Stockholm, Sweden •
 • Tel. +46(0)8 - 510 132 00 • Fax +46(0)8 - 655 35 10 •
 • www.humancaregroup.com •