HUMAN CARE

Your Life. Your Way.

Assembly Instructions / Monteringsanvisning

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Assembly Instructions

Art. nr: 10624

This assembly instruction explain how to mount the Slow Down Brake on the Carl-Oskar and Rebel rollators.

General information

The Slow Down Brake fits all widths and heights of the Carl-Oskar and Rebel rollators. It can also be mounted on Carl-Oskar RA and Walking Table, Rebel RA and Walking Table as well as on Carl-Oskar for heavy users and Rebel for heavy users.

Assembly Instructions

The Slow Down Brake is delivered in pairs, a brake for the right rear wheel and a brake for the left rear wheel. The knob on the side of the brake must be mounted on the outside of the rollator frame. (picture 1)

Unscrew the knob that holds it together the clamp. (picture 2)

Insert the loose cassette with the smaller wheel at the bottom of the Slow Down Brake. It will be loose until the brake is mounted on the rollator. The upper adjustment screws must be fully unscrewed before the Slow Down Brake is fitted (Fig. 3)

Open the clamp and ensure that the smaller cassette remains in the Slow Down Brake and that the smaller wheel in the cassette rests against the rollator's rear tire.
 (picture 4)

The distance between the lower edge of the Slow Down Brake and the upper part of the rear fork should be approximately 2 cm.
 (picture 5)

Tighten the knob on the side so that the brake is firmly attached to the frame and the small wheel rests against the rear tire, to get the desired resistance on the wheel. Adjust the hardness with the upper knob. Lock the upper knob with the wing nut. Test the rollator with the Slow Down Brake to ensure that the resistance is as desired.
 (picture 6)

Repeat this process on the other side.

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