

Full Body Sling & Full Body Sling Net



Full Body Net, 25005-x



Full Body, 25010-x



Divided Legs
Reclined Seating
Head Support



The Full Body sling is a comfortable sling with and without reinforcements, handles or padding in the back depending on which model you choose. They both offer good support for the entire back and extra support for the head.

Versatile Design

The Full Body slings are designed to create a reclined seated position which is extremely helpful if the patient has poor trunk stability and/or a tendency to overstress themselves (spastic performance) or are hypertonic. The reclined position prevents the user from sliding out of the sling. The reinforced head support makes this a suitable sling for lifting patients from a flat surface such as a floor.

Two Fabric Choices

Available in two different fabrics, the Full Body sling can be used in most lifting situations and fits most patients. With reinforced head support, this sling is suitable for lifting patients from flat surfaces such as floors. The Polyester model (shown) is made of low friction fabric that makes application fast and simple.

Like the Full Body sling, the Full Body Sling Net is a comfortable sling without reinforcements, handles or padding in the back and offers good support for the entire body. This sling can also be used in washing and bathing situations and can also be left behind the patient for short periods of time after the transfer has been completed.

Easy Care and Easy Fit

Our slings are manufactured to the highest quality standards. They feature reinforced loops and high strength nylon webbing around the sling to make them strong enough to resist even the most active use. Each sling label is colour coded to ensure quick and easy selection of the correct sling and the straps are colour coded for easy determination of required positioning.

Accessories



Extension Loops

Used to extend lift straps in order to alter lifting position or to hold leg parts together. Assorted lengths. Part no.:21180.*



Leg Padding and Supports

Ideal for users with sensitive skin or pain. Part no.: 21146-S, M, L Reinforced supports. One size. Part no. 25102



Upholstered Cover

Protects user from hanger bar impact. Not for use in a wet environment. Part no. 50866



Hanger Bars for 2-, 3- or 4-point Suspension

Part no. 57551, 55764 [2- or 3-Point - Quick Release], 55619 [4-Point - Attach to leg hook]

Lifting Situations



Use to/from a bed

Suitable for transfers and repositioning to and from a bed. Can be left behind patient for short period of time.



Use to/from a chair

Suitable for transfers and repositioning to and from a chair. Can be left behind patient for short period of time.



Use to/from the floor


Suitable for transfers from the floor.



Use for showering


Only Polyester Net fabric can be used for this application.

Care and Features

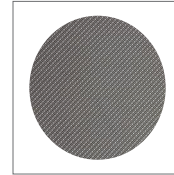


Polyester

Durable, low-friction, easy to clean, apply and remove.

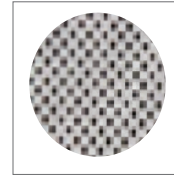

Polyester Net

Soft, thin material that breathes and dries quickly. Ideal for bathing or showering.

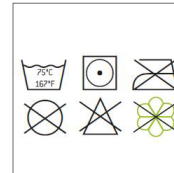

Care

Care instructions located on sling label. Use a hospital grade laundry bag while laundering.



Technical Specifications

Part Number	Size	Colour	Model #25005			Model #25010				
			A	B	C	A	B	C		
Polyester Net	Polyester									
25005-JM	n/a	Junior Medium		22cm / 8.7"	71cm / 28"	80 cm / 31.5"	-	-	-	<p>Model #25005</p>
25005-S	25010-S	Small		26cm / 10.2"	76cm / 29.9"	86cm / 33.9"	31cm / 12.2"	85cm / 33.5"	90cm / 35.4"	
25005-M	25010-M	Medium		26cm / 10.2"	85cm / 33.5"	95cm / 37.4"	40cm / 15.8"	92cm / 36.2"	91cm / 35.8"	
25005-L	25010-L	Large		34cm / 13.4"	96cm / 37.8"	95cm / 37.4"	40cm / 15.8"	98cm / 38.6"	100cm / 39.4"	
25005-XL	25010-XL	Extra Large		36cm / 14.2"	105cm / 41.3"	106cm / 41.7"	44 cm / 17.3"	105cm / 41.3"	102cm / 40.2"	
n/a	25010-XXL	Double Extra Large		-	-	-	44 cm / 17.3"	111cm / 43.7"	106cm / 41.7"	
n/a	25010-XXXL	Triple Extra Large		-	-	-	44 cm / 17.3"	118cm / 46.5"	108cm / 42.5"	
Maximum User Weight				300kg/661lb						<p>Model #25010</p>

* Available in assorted lengths: 10cm/3.9" [21180-10], 15cm/5.9" [21180-15], 20cm/7.8" [21180-20], 25cm/9.8" [21180-25], 35cm/13.7" [21180-35]