


Activity Sling



Head Support
Child/Teen Sizes
Shoulder Pads

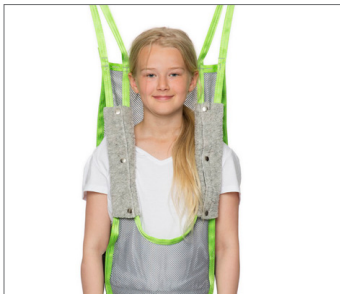

The Activity Sling is designed to help children or young teens with gait training or rehabilitation exercises. The Activity Sling is made of a Polyester Net and has an extended back side to support the neck and head while moving.



Comfortable

The Activity Sling features elasticized straps on the sides for maximum flexibility during movement. With added head support and comfortable shoulder padding, it allows for more comfortable and confident movement. The Activity Sling is designed for 2-point suspension and can only be used with an overhead ceiling lift.

elasticity which allows it to move with the patient and the “pant” style design ensures the sling will not slide up. The Activity Sling is applied in the supine position.

Easy Care and Easy Fit

Our slings are manufactured to the highest quality standards. They feature reinforced straps and high strength nylon webbing around the sling to make them strong enough to resist even the most active use. Each sling label is colour coded to ensure quick and easy selection of the correct sling and the straps are colour coded for easy determination of required positioning.



Versatile Design

The Activity Sling is designed to offer support for both the upper and lower body, however, the user must have some body control in order to move, balance and walk while doing gait training or rehabilitation exercises. The soft, breathable Polyester Net fabric has some

HUMAN CARE SWEDEN (HQ)

Årstaängsvägen 21B
117 43 Stockholm
Phone:- +46 8 665 35 00
Fax:- +46 8 665 35 10
info.se@humancaregroup.com

HUMAN CARE CANADA

10-155 Colonnade Road
Ottawa, ON K2E 7K1
Phone: +613.723.6734
Fax: +613.723.1058
info.ca@humancaregroup.com

HUMAN CARE NEDERLAND

Elspeterweg 124
8076 PA Vierhouten
Phone: +31 577 412 171
Fax: +31 577 412 170
info.nl@humancaregroup.com

HUMAN CARE UNITED STATES

8006 Cameron Road, Suite K
Austin, TX 78754
Phone: +512.476.7199
Fax: +512.476.7190
info.us@humancaregroup.com

Accessories



Extension Loops

Used to extend lift straps in order to alter lifting position or to hold leg parts together. Assorted lengths. Part no. 21180-*



Shoulder Padding

For added comfort. Comes with the sling. Replacement Part no. 25101



Upholstered Cover

Protects user from hanger bar impact. Not for use in a wet environment. Part no. 50866

Lifting Situations



Use to/from a bed

Suitable for transfers and repositioning to and from a bed. Can be left behind patient for short period of time.



Use to/from a chair


Suitable for transfers and repositioning to and from a chair. Can be left behind patient for short period of time.



Use for gait training/ mobilization


Suitable for gait training and rehabilitation exercises.

Care and Features

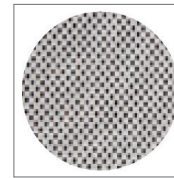


Polyester Net

Soft, thin material that breathes and dries quickly. Ideal for bathing or showering.

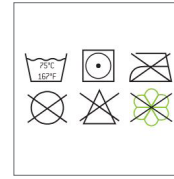

Care


Care instructions located on sling label. Use a hospital grade laundry bag while laundering.

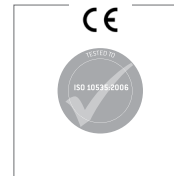

Tested to all relevant standards, including ISO 10535, hoists for the transfer of disabled persons.

Technical Specifications

Part Number	Size	Colour	Weight Range	A	B	C
Polyester Net			[Recommended]	[Open Sling]		
25100-JS	Junior Small	JS	15kg -20kg/ 33.1 lb - 44.1 lb	37cm / 14.6"	27cm / 10.6"	69cm / 27.1"
25100-JM	Junior Medium	JM	20kg -35kg/ 44.1 lb - 77.2 lb	39cm / 15.3"	30cm / 11.8"	75cm / 29.5"
25100-JL	Junior Large	JL	35kg - 50kg/ 77.2 lb - 110.2 lb	41cm / 16.1"	33cm / 13.0"	81cm / 31.9"
25100-S	Small (Teen)	S	40kg - 60kg/ 88.2 lb - 132.3 lb	43cm / 16.9"	39cm / 15.3"	87cm / 34.3"
Maximum User Weight		100kg/220lb				

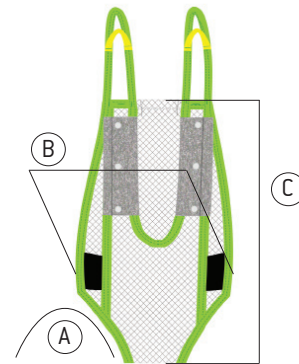

* Available in assorted lengths: 10cm/3.9" (21180-10), 15cm/5.9" (21180-15), 20cm/7.8" (21180-20), 25cm/9.8" (21180-25), 35cm/13.7" (21180-35)

This product is manufactured in accordance with all relevant ISO standards.

