

positioning slings



Model #NSB-410-ST
Ventilated Polyester



Model #NSB-401-ST
Ventilated Polyester



Model #NSB-451-U
Single Patient Use

Our positioning slings are available in reusable or single patient use models and are designed to assist a caregiver to safely and securely perform safe patient handling during sudden or intended positioning/repositioning, turning and lateral transfer with a patient in the supine position.

Ventilated Material

Our reusable ventilated slings are made of a breathable, pliable, material that has moisture-wicking properties. The unique material facilitates air flow making these slings ideal for critical care units as they can be placed behind the patient like a sheet for sudden and often unpredictable repositioning requirements. These slings easily fit most hospital beds and the material does not promote skin breakdown allowing the sling to remain behind the patient for short periods of time.

Single Patient Use

Our single patient use slings offer the convenient option of disposal after use with

the added benefit of hygiene control. Made of a durable Polyester/Cotton blend, these slings can be used by the patient for multiple lifts before disposal. Our single patient use slings have a weight capacity of 318kg/700lb.

Easy Care and Easy Fit

Our slings are manufactured to the highest quality standards. They feature eight reinforced straps (6 body/2 head) and high strength nylon webbing around the sling to make them strong enough to resist even the most active use. The straps are positioned to ensure optimal comfort and safety to the patient.

HUMAN CARE SWEDEN (HQ)

Årstaängsvägen 21B
117 43 Stockholm
Phone: +46 8 665 35 00
Fax: +46 8 665 35 10
info.se@humancaregroup.com

HUMAN CARE AUSTRALIA

Unit 4, 49 Bertie Street
Port Melbourne, VIC 3207
Phone : +61 1300 132 736
Fax : + 61 (03) 9645 4954
info.au@humancaregroup.com

HUMAN CARE CANADA

10-155 Colonnade Road
Ottawa, ON K2E 7K1
Phone : +613.723.6734
Fax : +613.723.1058
info.ca@humancaregroup.com

HUMAN CARE NEDERLAND

Elspeterweg 124
8076 PA Vierhouten
Phone: +31 577 412 171
Fax: +31 577 412 170
info.nl@humancaregroup.com

HUMAN CARE UNITED STATES

8006 Cameron Road, Suite K
Austin, TX 78754
Phone: +512.476.7199
Fax: +512.476.7190
info.us@humancaregroup.com

Accessories



Extension Loops

Used to extend lift straps in order to alter lifting position or to hold leg parts together. Assorted lengths. Part no.:21180.*



Hanger Bar for 4-point Suspension

70800 (4-Point - for HeliQ or HeliQ PT)



Hanger Bar for 4-point Suspension

55619 (4-Point - Attach to leg hook on Altair/ Roomer S)

Lifting Situations



Use to/from a bed

Suitable for transfers and repositioning to and from a bed. Can be left behind patient for short period of time.



Use to/from a chair


Suitable for transfers and repositioning to and from a chair. Can be left behind patient for short period of time.



Single Patient Use Model


Dispose after use. Intended for short term use by a single patient.

Care and Features

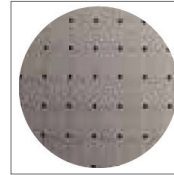


Ventilated Polyester

Maximum airflow and breathability. Can be left behind patient like a sheet.

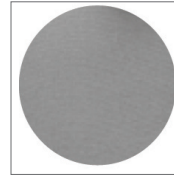


Polyester/Cotton

Single patient use.

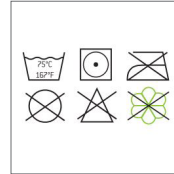

Care - Reusable Slings


Care instructions located on sling label. Use a hospital grade laundry bag while laundering.

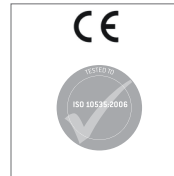

Tested to all relevant standards, including ISO 10535, hoists for the transfer of disabled persons.

Technical Specifications

Part Number	Size	Colour	Weight Capacity	A	B
Reusable Polyester, Ventilated					
NSB-401-ST	One Size	ONE SIZE	454kg/1000lb	107cm / 42.1"	198cm / 78"
NSB-410-ST-L	Large	L	454kg/1000lb	132cm / 52"	208cm / 81.9"
NSB-410-ST-XL	Extra Large	XL	454kg/1000lb	147cm / 57.9"	208cm / 81.9"
Single Patient Use Polyester/Cotton					
NSB-451-U	One Size	ONE SIZE	318kg/700lb	105cm / 41.3"	168cm / 66.1"
NSB-451-U ¹					

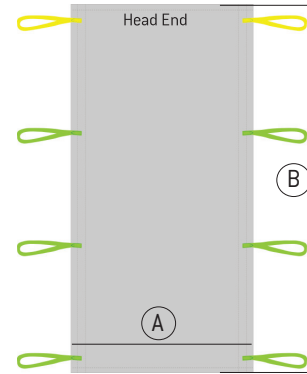

* Available in assorted lengths: 10cm/3.9" [21180-10], 15cm/5.9" [21180-15], 20cm/7.8" [21180-20], 25cm/9.8" [21180-25], 35cm/13.7" [21180-35]

¹ Only available in the United States

This product is manufactured in accordance with all relevant ISO standards.



Learn More About This Product



Follow Us on Social Media



facebook.com/humancareworld



instagram.com/humancaregroup



linkedin.com/company/human-care-ab



pinterest.com/humancaregroup



youtube.com/channel/UC7fjInh3zTjR0P1Df2Da0