

# Positioning Sling



Ventilated  
Bariatric Model  
SPU Option



Our positioning slings are available in reusable or single patient use models and are designed to assist a caregiver to safely and securely perform safe patient handling during sudden or intended positioning/repositioning, turning and lateral transfer with a patient in the supine position.

#### Ventilated Material

Our reusable ventilated slings are made of a breathable, plyable, material that has moisture-wicking properties. The unique material facilitates air flow making these slings ideal for critical care units as they can be placed behind the patient like a sheet for sudden and often unpredictable repositioning requirements. These slings easily fit most hospital beds and the material does not promote skin breakdown allowing the sling to remain behind the patient for short periods of time.

#### Single Patient Use

Our single patient use slings offer the convenient option of disposal after use with

the added benefit of hygiene control. Made of a durable Polyester/Cotton blend, these slings can be used by the patient for multiple lifts before disposal. Our single patient use slings have a weight capacity of 318kg/700lb.

#### Easy Care and Easy Fit

Our slings are manufactured to the highest quality standards. They feature eight reinforced straps (6 body/2 head) and high strength nylon webbing around the sling to make them strong enough to resist even the most active use. The straps are positioned to ensure optimal comfort and safety to the patient.

#### HUMAN CARE SWEDEN (HQ)

Årstaängsvägen 21B  
117 43 Stockholm  
Phone: +46 8 665 35 00  
Fax: +46 8 665 35 10  
info.se@humancaregroup.com

#### HUMAN CARE CANADA

10-155 Colonnade Road  
Ottawa, ON K2E 7K1  
Phone: +613.723.6734  
Fax: +613.723.1058  
info.ca@humancaregroup.com

#### HUMAN CARE NEDERLAND

Elspeterweg 124  
8076 PA Vierhouten  
Phone: +31 577 412 171  
Fax: +31 577 412 170  
info.nl@humancaregroup.com

#### HUMAN CARE UNITED STATES

8006 Cameron Road, Suite K  
Austin, TX 78754  
Phone: +512.476.7199  
Fax: +512.476.7190  
info.us@humancaregroup.com

## Accessories



### Extension Loops

Used to extend lift straps in order to alter lifting position or to hold leg parts together. Assorted lengths. Part no.:21180.\*



### Hanger Bar for 4-point Suspension

70800 (4-Point - for HeliQ or HeliQ PT)



### Hanger Bar for 4-point Suspension

55619 (4-Point - Attach to leg hook on Altair/ Roomer S)

## Lifting Situations



### Use to/from a bed

Suitable for transfers and repositioning to and from a bed. Can be left behind patient for short period of time.



### Use to/from a chair


Suitable for transfers and repositioning to and from a chair. Can be left behind patient for short period of time.



### Single Patient Use Model


Dispose after use. Intended for short term use by a single patient.

## Care and Features

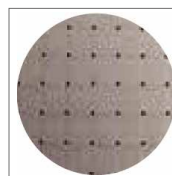


### Ventilated Polyester

Maximum airflow and breathability. Can be left behind patient like a sheet.

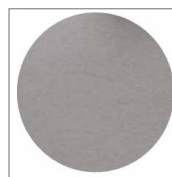


### Polyester/Cotton

Single patient use.

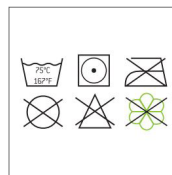


### Care - Reusable Slings

Care instructions located on sling label. Use a hospital grade laundry bag while laundering.

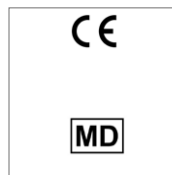

## Technical Specifications

Part Number	Size	Colour	Weight Capacity	A	B
<b>Reusable Polyester, Ventilated</b>					
NSB-401-ST	One Size	ONE SIZE	454kg/1000lb	107cm / 42.1"	198cm / 78"
NSB-410-ST-L	Large	L	454kg/1000lb	132cm / 52"	208cm / 81.9"
NSB-410-ST-XL	Extra Large	XL	454kg/1000lb	147cm / 57.9"	208cm / 81.9"
<b>Single Patient Use Polyester/Cotton</b>					
NSB-451-U NSB-451-A	One Size	ONE SIZE	318kg/700lb	105cm / 41.3"	168cm / 66.1"

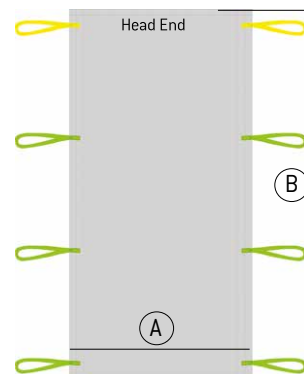

\* Available in assorted lengths: 10cm/3.9" (21180-10), 15cm/5.9" (21180-15), 20cm/7.8" (21180-20), 25cm/9.8" (21180-25), 35cm/13.7" (21180-35)

<sup>1</sup> Only available in the United States