

Columbus Sling E-clip Soft



Oval

Optimal support

Safe E-clips

Optimal support



Our Columbus Sling E-clip is an oval sling, making it ideal for people who need full support, even under their bottom. In certain specific situations, this sling can certainly prove to be an egg of Columbus.



This sling provides comfortable support to the head, torso, thighs and even the seat, helping the client feel safe and secure. This sling is ideally suited to clients with limited core stability and/or a different center of gravity due to, for example, an amputation or because they wear a corset. The design of the sling provides full support for the client, even under the buttocks, allowing the client to sit on the sling and eliminating leg slips. The seams in the Soft Air Mesh material fall next to the client's seat. This sling provides comfort and prevents pressure points giving the client the sense of optimal support.

point hanger bar. In a reclined position, clients with reduced core stability and/or a tendency to overstretch will not slip from the sling as easily. Thanks to the integrated, reinforced headrest, this sling can even be used to lift up clients from flat surfaces, such as the floor.

Soft Air Mesh™ is a lightweight, cool, soft and breathable fabric, with moisture-wicking properties, which means it draws moisture away from the skin through the fabric where it can evaporate quickly. Another advantage of this material is that it is anti-bacterial for improved hygiene. The sling has "two-way stretch" properties, making it feel almost as a second skin. Compatible with the mobile Diana II Comfort lift and Diana II 4 MSB.

Sitting back in a reclined position will reduce pressure on the client's leg, and this sling will help you do just that when used in combination with a four-



Mobility Aids



Healthcare Beds



Lifting Solutions



Hygiene



Convertible Chairs

Accessories



Spreader bar

For HeliQ stationary lift, available in Small (manual), Medium and Large (electric). Art. No. 92441, 92442 and 92443.



Spreader bar

For Diana II Comfort 4ESB (electric). Art. No. 90560.



Spreader bar

For Diana II/ Diana II Comfort 4MSB (manual), Art. no. 90558/ 90559.

Use



From and into bed

Suitable for lifting clients from bed and helping them sit down on bed.



From and to a chair


Suitable for positioning in a chair and for transfers in and out of a chair.



Lifting from the floor

Designed for lifts from the floor.

Features

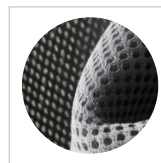


Soft Air Mesh

Great pressure-relief. Excellent pressure distribution thanks to the 3D-structure of the fabric. Prevents and protects.



E-clip integrated lock

Can be used one-handed. Universally compatible with all lifts equipped with mushroom fasteners Ø 6.8-8.2 mm. Ø 6.8-8.2 mm.

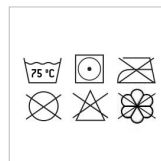


Cleaning and care instructions

For cleaning and care instructions, see the label of the sling. Wash in an approved laundry bag.

Technical specifications

Product name	Article number	Size	Colour	A	B	C
Columbus Sling E-clip Soft JS	25040C-JS	Junior Small	JS	42cm/16,5"	77cm/30,3"	124cm/48,8"
Columbus Sling E-clip Soft JM	25040C-JM	Junior Medium	JM	44cm/17,3"	80cm/31,5"	130cm/51,2"
Columbus Sling E-clip Soft S	25040C-S	Small	S	46cm/18,1"	84cm/33"	138cm/54,3"
Columbus Sling E-clip Soft M	25040C-M	Medium	M	53cm/20,9"	92cm/36,2"	143cm/56,3"
Columbus Sling E-clip Soft L	25040C-L	Large	L	57cm/22,4"	92cm/36,2"	153cm/60,2"
Columbus Sling E-clip Soft XL	25040C-XL	Extra Large	XL	69cm/27,2"	105cm/41,3"	156cm/61,4"
Columbus Sling E-clip Soft XXL	25040C-XXL	Extra extra large	XXL	69cm/27,2"	110cm/43,3"	160cm/63"
Maximum user weight		300 kg / 660 lbs				

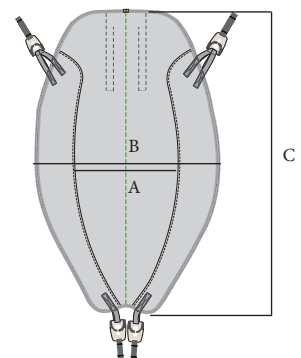

*We improve freedom of movement
and quality of care for people.*

www.humancaregroup.com



HUMAN CARE SWEDEN (HQ)

Årstaängsvägen 21B
117 43 Stockholm
+46 8 665 35 00
info@humancaregroup.com

HUMAN CARE CANADA

10-155 Colonnade Road
ON K2E 7K1 Ottawa
+1 613 723 6734
info.ca@humancaregroup.com

HUMAN CARE THE NETHERLANDS

Elspeterweg 124
8076 PA Vierhouten
+31 577 41 21 71
info.nl@humancaregroup.com

HUMAN CARE UNITED STATES

8006 Cameron rd, Suite K
TX 78754 Austin
+1 512 76 7199
info.us@humancaregroup.com