## User manual

Lift Trolley

Article number: 70346/70347

# Table of Contents

|                                         |        |
|-----------------------------------------|--------|
| ENGLISH.....                            | 2      |
| Introduction.....                       | 2      |
| Intended use.....                       | 2      |
| General features.....                   | 2      |
| Assembly instructions.....              | 3      |
| Technical details.....                  | 5      |
| Instructions for use.....               | 6      |
| Product labelling.....                  | 7      |
| Environment, transport and storage..... | 7      |
| <br>SVENSKA.....                        | <br>8  |
| Introduktion.....                       | 8      |
| Avsedd användning.....                  | 8      |
| Allmänna funktioner.....                | 8      |
| Monteringsanvisningar.....              | 9      |
| Teknisk data.....                       | 10     |
| Användningsinstruktioner .....          | 12     |
| Produktetikett.....                     | 13     |
| Återvinning.....                        | 13     |
| <br>ČESKY .....                         | <br>15 |
| Úvod.....                               | 15     |
| Určený účel.....                        | 15     |
| Obecné vlastnosti.....                  | 15     |
| Návod k montáži.....                    | 16     |
| Technické detaily.....                  | 18     |
| Návod k použití.....                    | 19     |
| Označení výrobku.....                   | 20     |
| Recyklace.....                          | 20     |

# ENGLISH

## ***Introduction***

Every single Human Care product is produced under high quality standards bearing in mind how to minimize the risk of environmental reverse effect. Human Care is certified by globally recognized authorities according to ISO 13485 and ISO 14001 standards.

Any incident involving the product should be reported to the manufacturer and a relevant regulatory body.

## ***Intended Use***



The Human Care Lift Trolley (70346) is produced with intent to be used in hospitals, nursing homes, and homes for transporting Human Care ceiling lifts (Roomer S and Altair) indoors. The ceiling lift shall be placed on the table and driven to where it is intended to be used.

The Human Care Lift Trolley (70347) is produced with intent to be used in hospitals, nursing homes, and homes for transporting Human Care ceiling lift, the Cosmos, indoors. The ceiling lift shall be placed on the table and driven to where it is intended to be used.

## ***General Features***


- Brakes on 2 rear castors
- Electrostatic coated metal frame
- Two table plates (one upper table top and one bottom plate)
- Push handle
- Two nobs on the right hand side of the table top's backside functions as holders for the charger and hand control holder. (only for 70346)

## ***Assembly Instructions***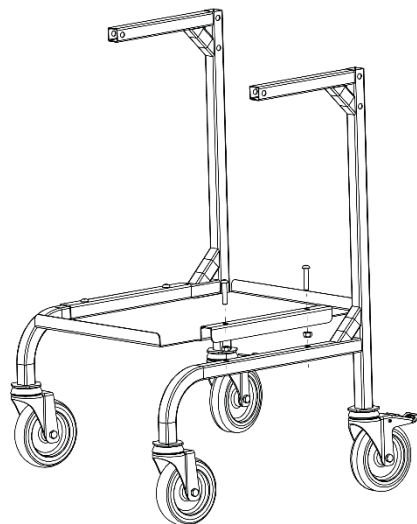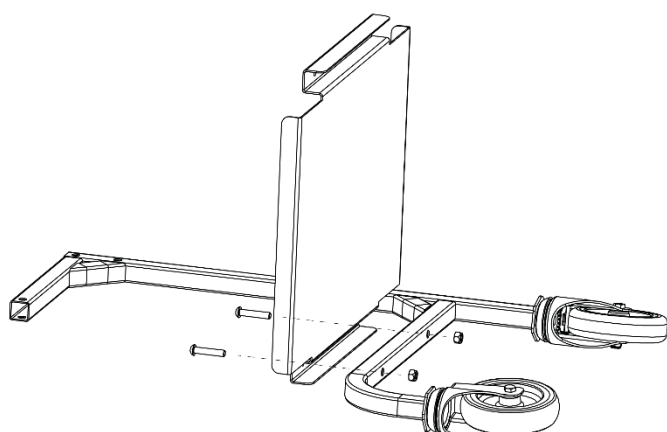

### **Step 1.**

- Fasten the bottom plate to one side of the frame as shown in the picture.
- Assemble the other side of the frame to the bottom plate.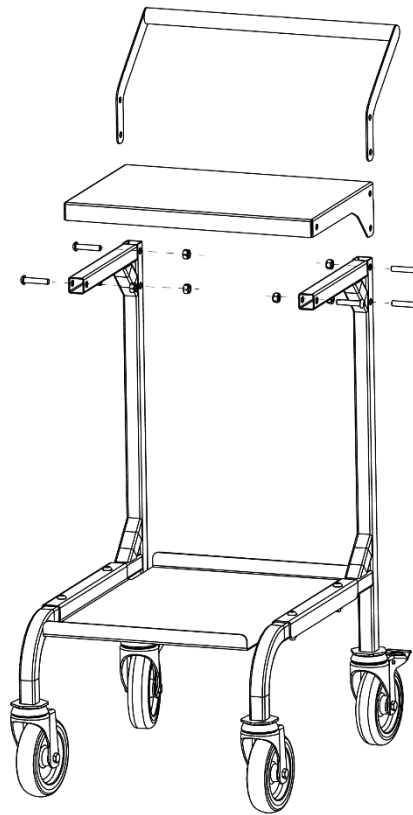

Step 2.

- Upper table top is placed on top of the side frames

Step 3.


- The handle bar is aligned with the holes and fixed with a bolt from the sides.

Step 4.


- After all the bolts are installed in order and the nuts are attached the bolts are held with the Allen key (B) and the nuts are tightened with the wrench (A).

## Technical Details

- 360 degree turning wheels, 2 of them with brakes
- Type and structure of paint: Electrostatic Powder Coat
- Upper tabeltop dimensions:  
275mm (art nr 70346), 329mm (art nr 70347) x 410mm
- Bottom plate dimensions: 480mm x 350mm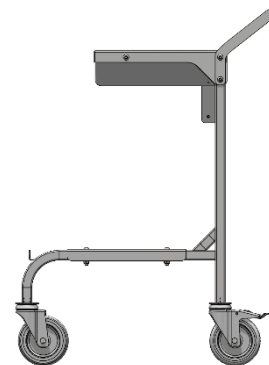

Art nr: 70347

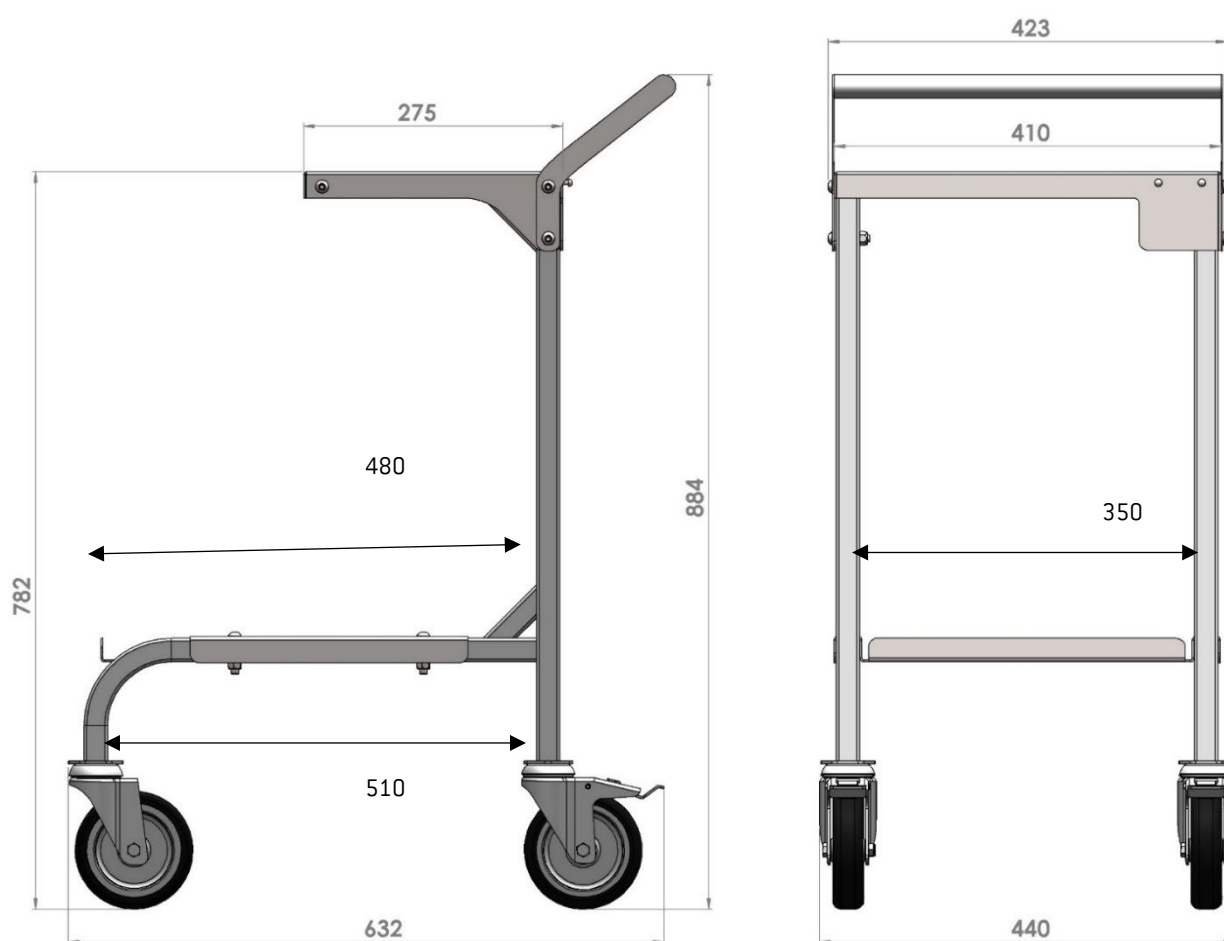Art nr: 70346

## ***Instructions for Use***

1. Place the ceiling lift on the upper table top with the two-point hanger bar hanging over the edge for Altair and Roomer S. See image 1a below.  
Place the Cosmos with the hanger bar hanging in between the top plates. See image 1b below.
2. Place the basket on the bottom table top. For instance, you can use the basket for transporting the sling you want to use.
3. Push the lift trolley to where the ceiling lift will be used.

Image 1a

Image 1b

# Product Labelling

## Environment, Transport and Storage

When recycling, the product's chassis is separated from other parts. At separation, the parts can be sorted according to material after unscrewing and dismantling rivets and other connections. The different parts should be separated into metal, plastic or combined materials. The products consist mainly of steel pipe and other metal components. Different authorities/recycling companies handle recycled materials in different ways. It is thus fitting when in any doubt over the best way to proceed when recycling, to contact the relevant governing body.

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