HUMAN CARE

Your Life. Your Way.

User Manual/Bruksanvisning/Notice
d'utilisation/Gebruikshandleiding

Gate System/Traversväxel/Jonction/Gate Systeem

Art. no. 55721

Contents

User Manual (EN)

Safety Instructions	3
Symbols	3
Warnings	3
Intended Use	3
General Care and Use	4
Technical Specifications	5
Understanding the Gate and Interlocking Systems	5
Components	6
Product Label	6
Product Symbols	7
Product Features	7
Installation	8
Maintenance	8
Troubleshooting	8

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Safety Instructions

Important!

You must read the user manual for your device prior to usage. Keep this booklet and information handy for future use.

Symbols

WARNING symbol indicates a potentially hazardous situation which, if not avoided, could result in serious injury to the user or damage to property and/or the device itself.

INFORMATION symbol indicates recommendations and information for proper, trouble-free usage.

WEIGHT CAPACITY symbol indicates maximum user weight for the product. Do not exceed this weight in any circumstance.

CE MARKING symbol indicates product meets requirements of EC directives.

RECYCLE packaging according to local regulations.

READ INSTRUCTIONS and ensure you fully understand them before using this product.

Warnings

DO NOT USE the product if,

Weight capacity is exceeded.

Product is damaged in any way.

You have not read and understood all warnings and instructions.

Intended Use

Gates enable transfers from a X-Y traverse system to a ceiling mounted rail, or the other way around in a safe manner (for example, the passage from a ceiling mounted rail to a traverse rail).

The Gate is an integral part of fixed and portable lifting rails used in health care, intensive care and rehabilitation.


Human Care's products are continuously being developed and updated and we reserve the right to make design changes without prior notice.

General Care and Use

Before using for the first time, make sure that:


You refer to the installation section in the manual 99946 while installing this product.

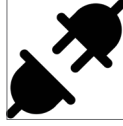
Disconnect the power supply during maintenance.


You have read and understood the user manual for the gate system and the interlocking mechanism.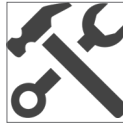

Product modification is not permitted and will void the warranty.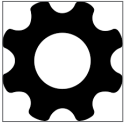


The gates and interlocking accessories are intended for use only with the gate system and must be correctly fitted to avoid risk of injury or damage.

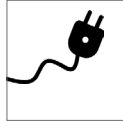Use only the supplied power supply.

Personnel who use the gate system have been given adequate training and understand its operation.

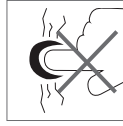Do not use fingers to activate sensors.

The right type, size, material and design of accessories have been selected by a professional with respect to the client's safety and needs.

To avoid the risk of electric shock, this equipment must only be connected to a supply main with protective earth.

Do not touch any parts which may have leakage current and the patient simultaneously (for example, parts of the plug in modules).




The gate system may only be used with Human Care products.

Technical Specifications

Description	Gate System	Interlocking System
Part no.	55721	
Dimensions	414mm x 232mm x 198mm 16.2" x 9.1" x 7.7"	432mm x 195mm x 147mm 17" x 7.6" x 5.7"
Product Weight	13kg / 28.7lb	
Safe Working Load	220kg / 458lb	
Power Supply	AC/DC Power Supply, ITE & Medical	
Input	100-240V~2.2A 50/60Hz	
Output	24V DC 7.5A	
Duty Cycle	10% Max 2 min ON/18 min off	
IP Rating	IP11	


Understanding the Gate and Interlocking Systems

The gate system includes a gate (for blocking/opening the rail) and an interlocking system (for locking functions). The gate system can only be used with Human Care rail installations. It connects using a 24V DC via an external power supply. Use only the power supply provided with the unit. The gates and interlocking mechanisms are driven by electrical motors and have two optical sensors, one located on each gate. Locking and unlocking of the gate and the interlocking unit can only occur when optical sensors are activated by a rail roller.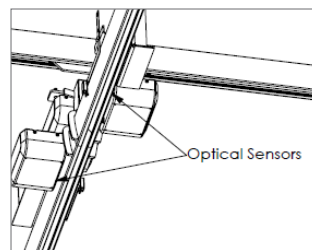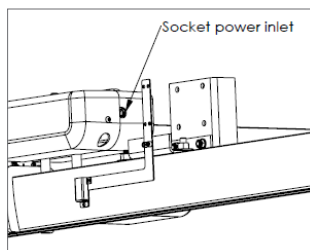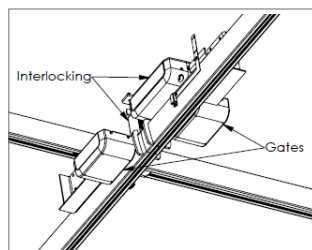

How the product works: Move the ceiling lift to the “gate end stop” and then align the traverse rail of the X-Y system. When the tracks are aligned properly, a beep indicates the next action is possible. The interlock opens and the ceiling lift can pass through onto the fixed track. When returning the ceiling lift to the traverse rail, the interlock automatically unlocks and secures the clamp at the ends.

Components

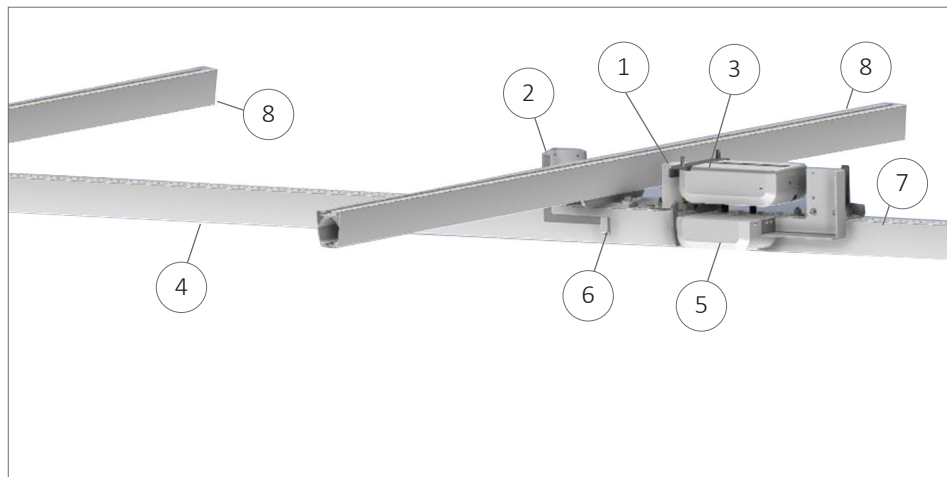- | | |
|----------------------|--------------------|
| 1. Interlock passive | 5. End Stop |
| 2. End stop | 6. Sensor End Stop |
| 3. Interlock active | 7. Fixed Rail |
| 4. Traverse Rail | 8. Secondary Rail |

Product Label

The product label contains information about the gate and interlocking system and is located on the interlocking unit.

This product has been tested by an accredited test institute and complies with the necessary requirements in accordance with medical requirements as follows: EN 10535, EN 60601-1 (Electrical Safety) and EN 60601-1-2 (EMC). This product is CE marked.

Product Symbols

Symbols on the unit and product label:

	CE Marking
	Date of manufacture
	Manufacturer
	Maximum load
	Instructions: Read the instructions and make sure you understand them before use
	Part number/reference number
	Serial number
	Warning
	The device contains electronics and it must be recycled appropriately
	Applied part type B: approved for physical contact with patients (IEC 60601-1)

Product Features

- Compatible with HeliQ, HeliQ-PT, Roomer S and Altair
- Compatible with our rails: P87, P120 and P180
- Smooth and silent operation, without any patient inconvenience
- Visible part of the products are in RAL-9016 colour (NCS0502Y)
- Entry and exit from a traverse system in a safe manner
- Logical and automatic in use
- Powered by mains
- Capable of handling a rail installation with more than one lift in operation
- Tested according to ISO 10535
- Operating time < 3 sec. (The cycle is from when the traverse is in correct position and the lift has entered the activation position to when passage is possible).

Installation

To mount a traverse system with a gate (55721), please follow the guide in our installation manual (99646). Only certified technicians are allowed to install the gate system. Please contact Human Care to get access to the document.

Maintenance

Annual inspection is required.

The Gates should be checked along with the lifting system.

Take special care to check any parts that may be exposed to wear and tear.

Cleaning

When cleaning the surfaces use an acid and chloride-free detergent, like warm water or a soap solution, with a soft cotton swab.

The cleaning sessions shall always end with a water rinse and drying with a cotton swab. The product must be dry before attaching to the lift.

Storage

Transport and store horizontally. The climatic conditions should be as follows:

Ambient Temperature from 0° to 40°C

Relative Ambient Humidity from 30% to 80%

Ambient Pressure from 790hPa to 1060hPa

Troubleshooting

Description	Action Required
Gates do not open when sensors are activated and a continuous beep sounds.	Two sensors are activated at the same time. Remove one rail roller from the sensor position.
Rail roller gets stuck by the gate sliding pin.	Wait 5 seconds and open gate. (Rail roller can then be removed).

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