HUMAN CARE
Your Life. Your Way.

Accessory manual

Fellow Classic, Fellow Supreme

Content

Accessory overview	3
Assembly Tray	4
Assembly Basket	5
Assembly Slow Down Brake	5
Assembly Newspaper Strap	5
Assembly cane holder	6
Assembly Bag	7
Assembly Back Support	7
Assembly Curb Climber	8
Assembly IV-stand	8
Assembly Oxygen Holder	10
Assembly One Hand Brake	11

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Fellow Classic, Fellow Supreme

Accessory overview

	1 Tray Practical for resting items. Part no, Fellow Classic: 08806 (image 1)		2 Part no. Fellow Supreme: 10667 (Image 2)
	1 Basket with handle Basket ideal for shopping. Part no, Fellow Classic standard: 08805. Fellow Classic low: 88052. Fellow Classic narrow, wide and high: 88052. (Image 1)		2 Part no Fellow Supreme: 10666 (Image 2)
	Slow down brake Prevents the rollator from moving too fast. Part no: 10625 (set)		
	Cane holder Simplifies the transportation of a cane. Part no: 10603		
	Soft carry case ideal for personal items. Part no: 08808. (Cannot be used on Fellow Classic low)		
	Back rest Gives support for the back when sitting. Part no: 10668		
	Curb climber Simplifies the transport over obstacles. Integrated in rollator for Fellow Supreme. Part no (Fellow Classic):10661		
	IV-stand Can only be used with Fellow Classic. Part no: 89654		
	Oxygen holder Part no, Fellow Classic: 08811. Part no. Fellow Supreme: 10671		
	Newspaper strap Can only be used on Fellow Supreme. Part no. 10679.		
	One-hand brake Allows control of brakes with one hand. Part no, Fellow Classic, right: 10677, left: 10678. Part no. Fellow Supreme, right: 10669, left: 10676.		


Attaching the Tray

ENG

Fellow Classic


1. Attach the bracket of the tray over the back edge of the seat.
2. Secure the tray by firmly pressing it over the front edge of the seat.

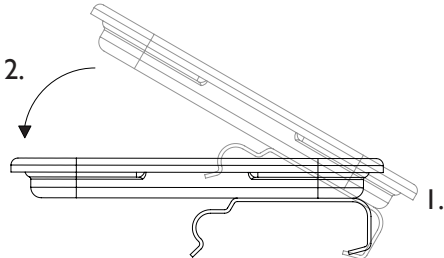Lift the tray in the front edge when removing it.


Fellow Supreme

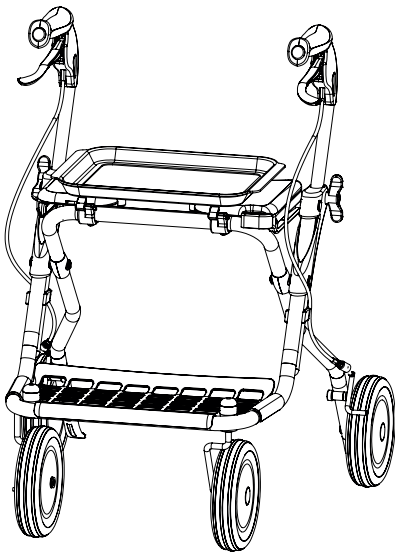
Assemble the tray on to the seat of the rollator as according to picture below.

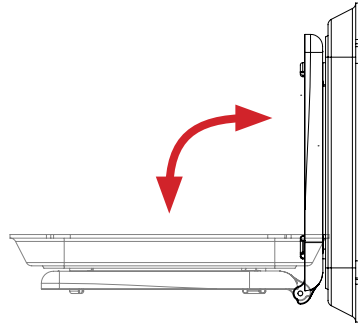1. Remove the tray by unfolding the seat.

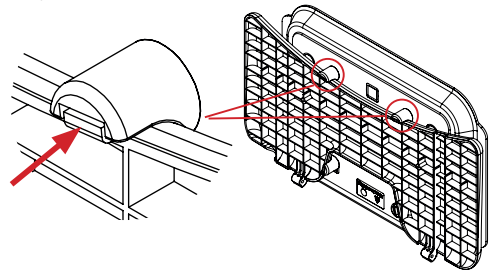Push underneath the tray's feet to remove the tray from the seat.

Attaching the Basket

ENG

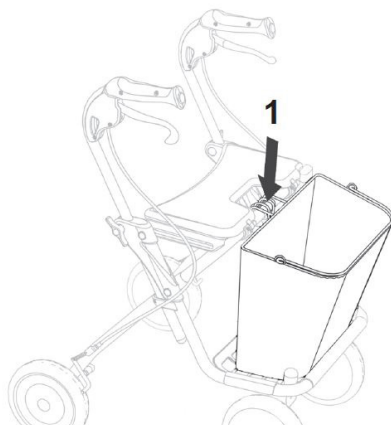
Fellow Classic

Hang the basket on the pipe to the locking device and place it on the grating. Max load 10kg.

Fellow Supreme

Place the basket on the grid and push the bracket over the frame.

Attaching the Slow Down Brake

ENG

Slow down brake - right and left

The trailer brake is assembled by pushing it onto the brake gear and fasten it through leading the hook on the underside of the brake gear.

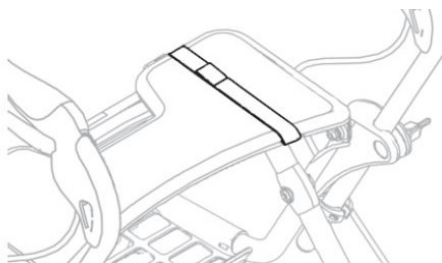
Adjust the roll resistance by screwing the adjusting screw. Lock the adjusting screw with the stop nut.

Attaching the Newspaper Strap

ENG


Pull the news paper strap over the seat to the intended track. Only for Fellow Supreme


Assembling upper part.

The cane holder is delivered in two main parts, an upper and a lower. These are in turn composed of two parts, verify that you have received them.

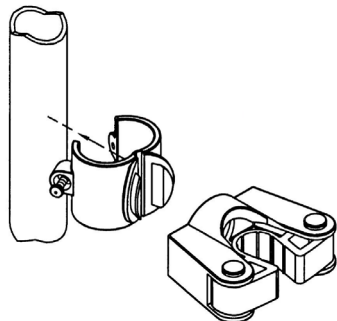
1. Open the clamp about that much it can be lead over the handle pipe.

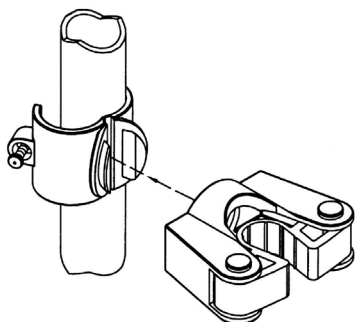2. Before closing the clamp, take the attachment and lead the pliers of the clamp through the hole of the attachment. Attend on turning it in preferred position.

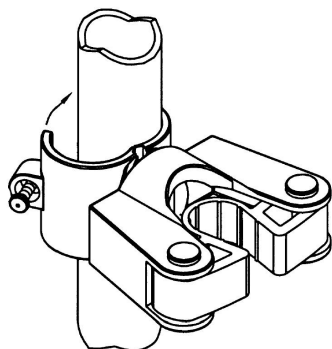
3. Close the clamp and tighten the screw. Note! :ured.


Assembling lower part.

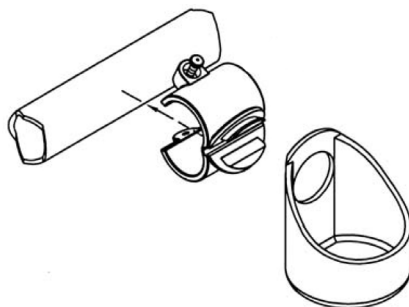
1. Open the clamp about that much it can be lead over the handle pipe.

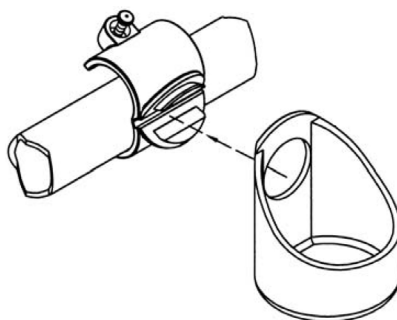2. Before closing the clamp, take the cup and lead the pliers of the clamp through the hole of the attachment. Attend on turning it in preferred position.

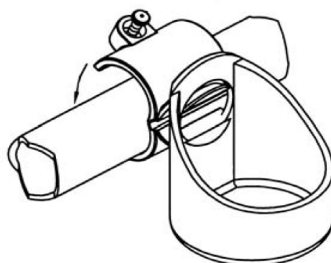3. Close the clamp and tighten the screw. Note! Make sure that the cup is secured.

Attaching the Bag

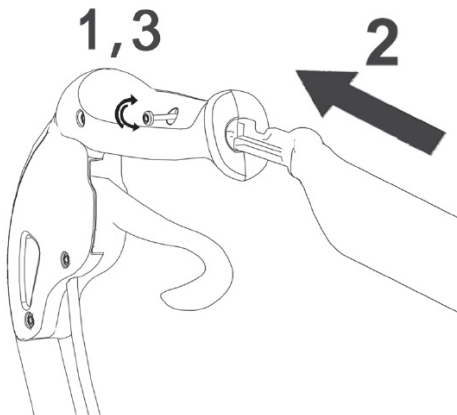
ENG

Form adjusted carrier bag that should be adjusted around the pipe at the hold of the seat.

Assembly Back Support

ENG

1. Loosen the screws and remove the knobs on both sides.
2. Insert the back support into the end of the handle.
3. Tighten the screws and the back support is assembled.

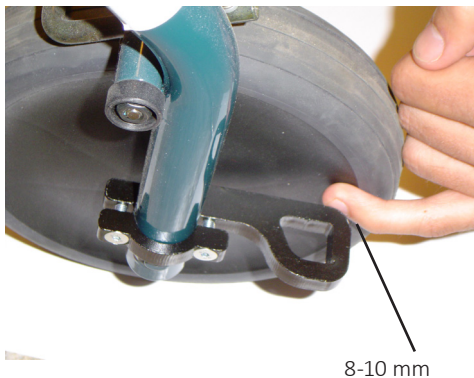
The rollator can NOT be suspended, carried or lifted in the back support.

Make sure the back support is securely fastened. Pull the back support to make sure it is fastened properly. **NOTE! Pay attention to cracks or loose parts.**

Assembly Curb Climber

ENG

Only for Fellow Classic

Assembly IV-stand

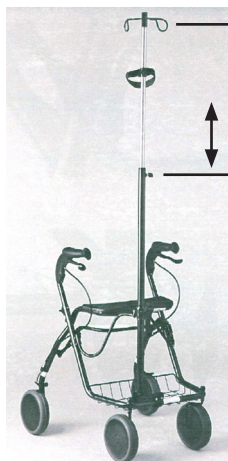
ENG

Product description

The drip stand consist of following parts:

- Clamps, 3 pc
- Bottom support
- Pipe (lower) - including lock housing, screws and handle
- Bended side pipes (right, left)
- Pipe (upper) with holder for drip - including velcro.
- Counterweighth
- Bundle straps for counterweight, 2 pc.
- Allen key 5 mm.

Only for Fellow Classic

The IV bag/bottle is to be hanged here.

The heigth is adjusted by loosening this screw, heightning or lowering the upper part, and tightening the screw.

Safety information

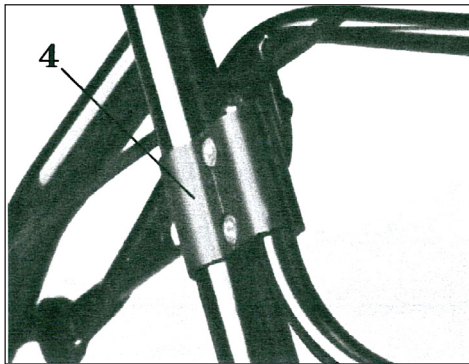
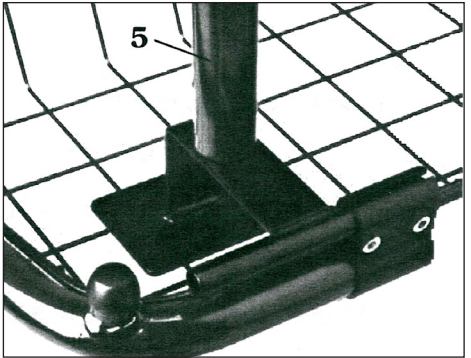
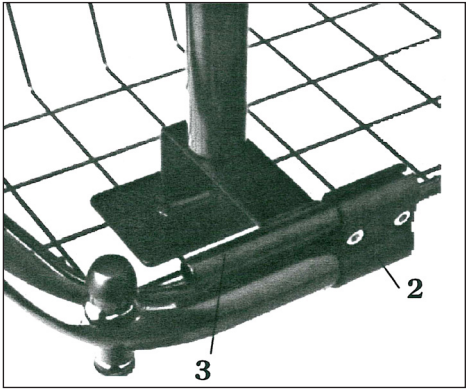
Important for technicians, prescribers and users to know

- Rollator with IV stand must only be used on a plane level.
- Point the hooks on the IV stand towards the middle of the rollator to avoid skew load.
- Maximum load on the IV stand is 3 kg.
- Secure the intravenous drip bag/bottle with the velcro.
- The IV stand must not be used as a support when getting up.
- The IV stand can only be used with till Fellow standard, wide and low.


Assembling

1. Choose on which the side drip stand should be assembled on.
2. Assemble a clamp between the pipe of the front part and grating (screws to the outside and clamp upwards).
3. Attach the bottom support to the clamp. Rough adjustment - the attachment pipe of the bottom support should be assembled about 1 cm from the protection cover of the bearings.


4. Assemble the other two clamps on the frame of the front part under the lock pegs (the screws pointing outwards and the clamp straight forward).
5. Put down the pipe in the hole of the bottom support.

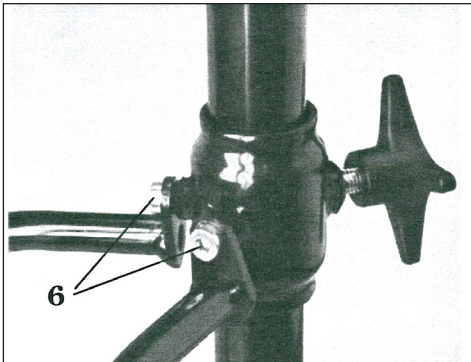6. Assemble the bended side pipes in the lock housing. Make sure respectively side pipe is placed on the right side.

Note! Do not tighten firmly.

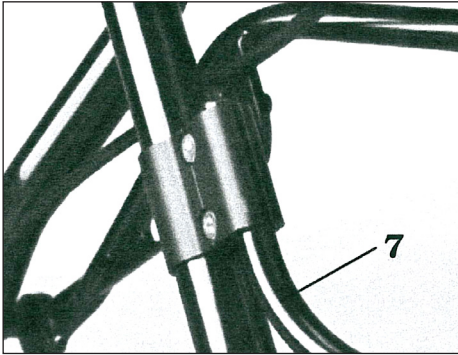7. Loosen the handle on the lock housing.

8. Put the ends of the side pipes in the clamps from below.

Note! The pipe ends must match side to side with the clamp.

8. Tighten firmly.

9. Adjusting to get a vertical dripstand: Achieve to get the upper clamps to the same height, pointing forward. Then loosen a little on the lower clamp, turn it outwards or inwards depending on preferred angle. There after move the clamp sideways to achieve the right sideways angle. When adjustment is ready tightening all screws and clamps.

Note! Maximum heightening of the drip stand is when the upper part of the white mark is getting visible

10. If the drip stand is placed on the right side: Place the counterweight at the left side, as close to the side as possible, and secure it with the two bundle straps.

If the drip stand is placed on the left side: Place the counterweight at the right side, as close to the side as possible, and secure it with the two bundle straps.

The function of the counterweight is to give a better balance to the rollator and to reduce the tilt risk.

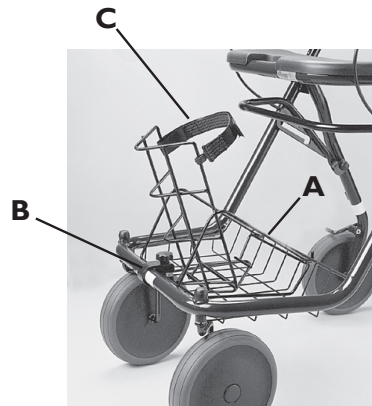
Assembly Oxygen Holder

ENG

Fellow Classic

1. Place the oxygen holder on the grating of the rollator.
2. Attach the two hooks on the rear end of the grating, A.
3. Attach the basket with a plastic clamp around the front frame of the rollator, B.
4. Tighten with enclosed screw and knob.
5. Fix the oxygen tank with the velcro at the oxygen holder, C.

Fellow Supreme

1. Remove the grating and put the oxygen tank holder on the frame.
2. Tighten the oxygen tank holder on the frame with the strap.
3. Assembly the tank on the oxygen tank holder with the velcro.

Note! Maximum load 5kg

Assembly One Hand Brake

ENG

The kit consists of following parts:

- A complete pre-mounted handle with brake lever and handlebar.
- wires
- casings
- distribution piece
- spring
- end seals
- wire lock

Disassemble before mounting single-hand brake

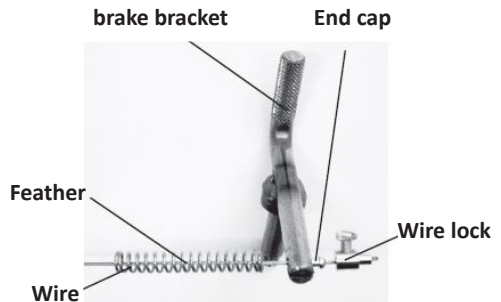
1. Remove the wires by the wheels on both sides of the rollator.
2. Remove the wire and lever on the side that will not to be used as a single-hand brake
3. Remove the handle tube on the side of the handbrake.

Mounting of one-hand brake

1. Mount the one-hand brake on the desired side. The distribution piece with the two holes up at the handle is rotatable. Make sure that the wires are not crossed, ie. The long wire should be mounted to the center of the rollator.
2. Pull down the covers and the wires to the wheels, the same way as the demounted sets.

3. Step through each wire through cover, wire bracket, spring and brake bracket. Step on the end sleeve with the knob down to the floor, the sleeve should be in the bracket bell hole and prevent the wire from jumping, see also the illustration.

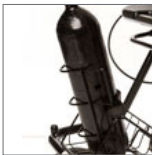
Step on the wire lock and adjust the brake position so that it will work well. Check that both ends of the housing are inserted into the correct position. Tighten the cable lock and cut the remaining wire.

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