PM-8091

PM-8090

Fußstuhl Footstool

Gebrauchsanleitung · Manual
Bruksanvisning · Käsikirja · Brugsanvisning
Instrukcja obsługi

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1 Allgemeine Informationen

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Table of Contents

1 General Information

- 1.1 Product versions
- 1.2 Accessories
- 1.3 Scope of delivery
- 1.4 Storage, transport
- 1.5 Service life / lifecycle

2 Safety information

- 2.1 Symbols used
- 2.2 Function check

3 Using the product

- 3.1 Indication and further use
- 3.2 Contraindications
- 3.3 Application risks
- 3.4 Areas of application
- 3.5 Application illustration
- 3.6 Application
- 3.7 Approved accessories, combination with other products

4 Product marking / CE

5 Cleaning, care and disinfection

6 Technical Data

7 Warranty

8 Disposal information

1 General Information

Thank you for choosing a Petermann product.

Please read the user manual carefully before using the product for the first time!

The Foot Stool raises the standing surface. The Foot Stool is used to bridge height differences and for stabilisation during the transfer. In hospitals and other health care facilities, as well as in home care, the Foot Stool is used to increase safety for all involved persons.

The use of the Footstool Cover improves hygiene conditions and simplifies cleaning.

The product may only be used for the specified purposes and conditions.

The information in this user manual is for the patient as well as for the use.

Contact us if you require additional information or assistance using the Petermann product.

1.1 Product versions

Item no.: PM-8090 Footstool (9 cm)

Item no.: PM-8095 Footstool (5 cm)

1.2 Accessories

Item no.: PM-8091 Footstool Cover (9 cm)

Item no.: PM-8096 Footstool Cover (5 cm)

1.3 Scope of delivery

Footstool
User manual

Footstool Cover
User manual

1.4 Storage, transport

Store/transport at -5 C – 40 C
Other: protect from sunlight

1.5 Service life / lifecycle

If the product is used in accordance with the safety instructions, then the expected product service life is **five years**. However, the effective service life may vary depending on the frequency and intensity of use and the number of cleaning and disinfection cycles.

Perform regular function checks on the product and be sure to observe the safety information (see item 2).

2 Safety information

WARNING indicates a potential danger which could lead to personal injury or material damage.

- It is the responsibility of the organisation / the user to determine, on the basis of the circumstances (weight / health condition of the patient) and the capabilities of the user, whether several users may be required for the transfer and/or whether additional safety measures are required.
- Defective products may no longer be used.
- Any use of the Footstool other than the intended use is prohibited!
- Ensure that the transfers are performed by the users in a back-friendly manner.
- Ensure that the patient's feet are positioned correctly and fully on the Footstool.
- Ensure that the Footstool is positioned securely on the floor.
- Ensure that the Footstool isn't an obstacle in the room and that nobody can stumble over it.
- Weight limits must be observed (200 kg/440 lb).
- Don't use any pointy objects in combination with the Footstool, since this could result in damage to the product.
- Positioning aids are subject to a natural ageing process. Inspect the equipment regularly to ensure proper functionality.
- The use of non-standard cleaning agents or bleaching agents, alcohols, other chemical additives may lead to permanent deterioration of the Footstool properties.

2.1 Symbols used

	Wipe off		max. load capacity see item 6
	Do not bleach		Manufacturer
	Do not tumble		Batch no. / lot no.
	Do not iron		Identification code
	Do not dry clean		Conformity mark
	Warning		Item number
	Observe the user manual		Medical device label
only valid for Cover Footstool			
	60 C laundry		

2.2 Function check

Transfer products are also subject to a natural ageing process. Inspect the equipment regularly and carefully, in particular before each use.

Visual inspection

Ensure that material, seams, handle loops and buckles are undamaged and in functional order.

- **Defective products may no longer be used!**
- **Product modification is not permitted!**

3 Using the product

3.1 Indication and further use

Due to the wide range of situational support options, the indications cannot be tied to a specific clinical picture, since positioning aids are suitable für all applications involving people requiring support.

This product may be reused.

Before reusing the product, be sure to perform a thorough function check (see item 2.2) as well as cleaning or disinfection of the product (see item 5).

3.2 Contraindications

Contraindications cannot be tied to a specific clinical picture. Under consideration of the safety information (see item 2), the product may not be used if the patient has any of the following conditions: The patient can not put any weight on their feet.

- The patient is too weak
- The patient has severe/frequent dizzy spells
- Extension tone during transfer
- The Patient has insurmountable fear during transfers.

3.3 Application risks

- Uncontrolled mobile or active patients → risk of falling
- No weight is placed on the feet / the Footstool → The Footstool slides away → risk of falling
- Footstool remains on the floor → Risk of tripping
- Incorrect assessment of the patient's abilities → risk of falling

3.4 Areas of application

The product can be used in home care and other health care facilities as well as hospitals.

3.5 Application illustration

3.6 Application

3.6.1 Footstool

The support possibilities of the Foot Stool serve to activate or secure the patient and provide the user with a stable support surface for the transfer of active or immobile patients. Pressure on the underside of the thighs, which can occur when sitting on the bed or chair due to the side rails or the edge of the chair, is reduced by using the Foot Stool. The Foot Stool can be used to position the feet for a better stability when sitting.

The Foot Stool has anti-slip properties when under load and can easily be moved to the desired position when not under load.

3.6.2 Footstool Cover

Pull the Footstool Cover over the Footstool and close the zipper. There is a handle on the top of the Footstool Cover for carrying. The Footstool Cover can be washed at 60° without the Footstool.

Visit our website at www.pm-med.de or watch our comprehensive product videos on our YouTube channel.

3.7 Approved accessories, combination with other products

The Footstool can be used in combination with the following products:

- PM-8091 Footstool Cover
- PM-3005 Leg Belt → stabilisation and safety
- PM-5020/30/50 Sliding Board → seated transfer
- PM-6013/14/15 Support Belts → safety
- PM-5080 Turning Disc → to facilitate the transfer

Petermann transfer aids are part of a comprehensive aid product range, which are effective and back-friendly for the user and comfortable and supportive for the patients. Additional Petermann products may be used according to the situation and clinical picture. Ask our experts. We are happy to help you!

4 Product marking / CE

The Petermann Footstool is a Class I Medical Device and meets the EU-Regulation 2017/745 of the European Parliament and Council dated 05.04.2017 for medical devices, also called MDR.

Petermann GmbH is certified in accordance with DIN EN ISO 13485 by TÜV Süd, Medical Devices - Quality management systems applicable to the design and development, production and distribution of aids for people of limited mobility.

5 Cleaning, care and disinfection

Cleaning	Desinfection (if necessary)
Damp cleaning	Recommended disinfectants for wipe disinfection: <ul style="list-style-type: none"> - Trionic® (DGHM/VAH approval, further information on request) - Mikrobac® Tissues* - Mikrobac® forte* - BacilloI® 30 Foam - BacilloI® 30 Tissues - BacilloI® 30 Sensitive Tissues - Mikrobac® Virucidal Tissues*
Warm water / mild detergent	
60° washing machine (only Footstool Cover)	
Mild detergent (only Footstool Cover)	
Do not use cleaning agents with bleaching ingredients or chemical laundry additives! Otherwise the product properties may deteriorate permanently.	
Wash with similar colours only	* wipe with a cloth soaked in water after disinfection to avoid residues or layer build-up. Please always follow the manufacturer's manuals for use when wiping disinfection.
Recommended detergent for industrial washing machines:	
- - Maximo Liquid as single detergent	
Low dryer temperatures (45° - 50° C)	

Be sure to follow the care instructions on the label of each product as well as the manuals for use.

6 Technical Data

Item no.	Material	Colour	Load-bearing capacity
PM-8090/PM-8095	Polyethylen, latex free	blue	200 kg/440 lb
PM-8091/PM-8096	Polyamide, latexfree	grey	200 kg/440 lb

Item no.	Description	Dimensions in mm/"	Weight	UDI
PM-8090	Footstool	380 x 320 x 90 14,9 x 12,6 x 3,9	480 g / 1 lb	(01)04250355380906(10)x
PM-8095	Footstool (5 cm)	380 x 320 x 50 14,9 x 12,6 x 2,2	270 g / 0,56 lb	(01)04250355380951(10)x
PM-8091	Footstool Cover	380 x 320 x 90 14,9 x 12,6 x 3,9	110 g / 0,24 lb	(01)04250355380982(10)x
PM-8096	Footstool Cover	380 x 320 x 50 14,9 x 12,6 x 3,9	60 g / 0,12 lb	(01)04250355380968(10)x

The manufacturing of our products is subject to highest quality standards. Due to usual manufacturing variations published dimensions and further information concerning the condition of the product are defined as target production specifications. Deviations which might occur are monitored by professional quality management procedures and do not represent a reduction in performance of the respective product.

7 Warranty

- The company Petermann warrants that the product is free from defects in material and workmanship.
- The warranty period is 3 years. **It does not cover defects attributable to abrasion, normal wear, improper use or improper cleaning.**
- The company Petermann is not liable for accidental or consequential injury or material damage.

8 Disposal information

There are no specific disposal regulations for Footstools, they can be disposed of in accordance with applicable national and local regulations.

Petermann GmbH
Waldweg 3
D-91601 Dombühl
Phone: +49 (0) 9868-9339-0
Email: petermann@pm-med.de

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