Spannbettuch Fitted Sheet

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Table of Contents

1 General Information

- 1.1 Product versions
- 1.2 Accessories
- 1.3 Scope of delivery
- 1.4 Storage, transport
- 1.5 Service life / lifecycle

2 Safety information

- 2.1 Symbols used
- 2.2 Function check

3 Using the product

- 3.1 Indication and further use
- 3.2 Contraindications
- 3.3 Application risks
- 3.4 Areas of application
- 3.5 Application illustration
- 3.6 Application
- 3.7 Approved accessories, combination with other products

4 Product marking / CE

5 Cleaning, care and disinfection

6 Technical Data

7 Warranty

8 Disposal information

1 General Information

Thank you for choosing a Petermann product.

Please read the user manual carefully before using the product for the first time!

The current understanding in modern occupational health and safety is that people in nursing professions experience significant work-related physical strain. Back injuries are common. There are many situations in which a patient has to be transferred from a bed, a wheeled stretcher, ambulance stretcher, an X-Ray table or a wheelchair to a different stretcher or seat. The strain on the user and the patient / care-dependent can be reduced significantly through the use of sliding mats.

The use of transfer aids should be conditional upon the user having learned to use the equipment properly. The user has to decide for themselves what equipment is suitable to the patient's situation and the intended sequence of movements.

The Fitted Sheet is part of a product line which is used in combination with the Turn Sheet specifically to aid in the repositioning and/or transfer of patients in home care and other health care facilities.

The product may only be used for the specified purpose and under the specified conditions.

The Fitted Sheet in combination with the Turn Sheet is a safe repositioning aid, which allows for repositioning which is pleasant and comfortable for the patient as well as being simple and back-friendly for the users.

The information in this user manual is for the patient who is being transferred and moved, as well as the user performing the transfer / position change.

Contact us if you require additional information or assistance using the Petermann product.

1.1 Product versions

Item no.: PM-1584

1.2 Accessories

No accessories are available for this device.

1.3 Scope of delivery

Fitted Sheet
User manual

1.4 Storage, transport

Store/transport at -5°C – 40°C
Other: protect from sunlight

1.5 Service life / lifecycle

If the product is used in accordance with the safety instructions, then the expected product service life is **five years**. However, the effective service life may vary depending on the frequency and intensity of use and the number of washing and disinfection cycles.

Perform regular function checks on the product and be sure to observe the safety information (see item 2).

2 Safety information

WARNING indicates a potential danger which could lead to personal injury or material damage.

- It is the responsibility of the organisation / the user to determine, on the basis of the circumstances (weight / health condition of the patient) and the capabilities of the user, whether several users may be required for the transfer and/or whether additional safety measures are required.
- Defective products may no longer be used.
- The smooth surfaces significantly reduce resistance due to friction during repositioning. Check the transfer surfaces carefully to ensure they are fixed in place securely (e.g. wheelchair, bed).
- Ensure that the transfers are performed by the users in a back-friendly manner.
- In case of direct skin contact, observation of changes in skin appearance is indicated.
- Weight limits must be observed (200 kg).
- Transfer products are subject to a natural ageing process. Inspect the equipment regularly to ensure proper functionality.
- The use of non-standard cleaning agents or bleaching agents, alcohols, fabric softeners or other chemical laundry additives may lead to permanent deterioration of the sliding properties.

2.1 Symbols used

| | | | |
|---|-------------------------|---|-------------------------------|
|  | 90°C laundry |  | max. load capacity see item 6 |
|  | Do not bleach |  | Manufacturer |
|  | Dry at low temperatures |  | Batch no. / lot no. |
|  | Do not iron |  | Identification code |
|  | Do not dry clean |  | Conformity mark |
|  | Warning |  | Item number |
|  | Observe the user manual |  | Medical device label |

2.2 Function check

Transfer products are also subject to a natural ageing process. Inspect the equipment regularly and carefully, in particular before each use. Ensure that the product is in sound condition and that the material and the seams are not damaged. Check the product functionality and ensure that the material withstands the load. The smooth surface of the special sliding material allows for easy sliding and moving. The rough surface of the anti-slip material prevents slipping. Check these properties - ideally in comparison to a new product.

➤ **Defective products may no longer be used!**

3 Using the product

3.1 Indication and further use

Due to the wide range of possible applications, the indications cannot be tied to a specific clinical picture, since the Fitted Sheets in combination with the turn sheet are suitable for all applications involving people who require assistance. Thanks to the reduced friction, the capabilities of the patient can be put to use when Fitted Sheets are used. The effort required on the part of the user is reduced. The product can be used in combination with the Turn Sheet in support of the following diagnoses.

- For poor skin condition
- For large area wounds
- With decubitus
- For patients with an increased risk of bleeding due to skin contact
- For micro position changes
- For individually felt pain
- To the transfer of overweight patients
- As an option for recumbent transfer from bed to an adaptive wheelchair
- For mobilisation and positioning of a patient in bed according to indications

- For patients with elevated or reduced muscle tone
- For patients unable to move on their own
- Can be used with minimal independent patient mobility (e.g. ALS or MS)
- To reduce shear and friction forces
- Application in palliative settings
- A combination of pressure relieving mattresses, pain treatment systems or alternating pressure mattresses is possible in specific configurations after testing.

This product may be reused.

Before reusing the product, be sure to perform a thorough function check (see item 2.2) as well as cleaning or disinfection of the product. (see item 5)

3.2 Contraindications

Contraindications cannot be tied to a specific clinical picture. Under consideration of the safety information (see item 2), the product may not be used if the patient has any of the following conditions:

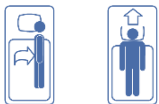
- Independent/active patients
- Wandering patients

3.3 Application risks

- Patients are miscalculated → risk of falling
- The product is embedded incorrectly → risk of falling

3.4 Areas of application

The product can be used in home care and other health care facilities as well as hospitals.

3.5 Application illustration

3.6 Application

The Fitted Sheet is an accessory item for the Turn Sheet (item no. PM-1582) or the Turn Sheet with hidden handles (item no. PM-2098). The sliding surface of the fitted sheet is finished with extremely low-friction sliding material. The fact that the sliding surfaces of the top sheet and the turn sheet lie against one another maximises the reduction of friction forces, producing the same effects as a sliding mat.

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When using the Fitted Sheet in combination with the turn sheet, anti-slip mat and a 3-piece bed wedge set (item no.: PM-1695), the following parts are used:

- Turn Sheet 3 belt loops at the head end and 12 hidden handle loops
- Fitted Sheet with sliding surface and anti-slip safety strip
- Anti-Slip Mat and support belts to aid with supporting and positioning
- Bed Wedges for positioning the patient

3.6.2 Preparation

- Fit the Fitted Sheet onto the mattress like a fitted bed sheet with the sliding surface facing up.
- Tighten it on the sides using the pull cords.
- Place the Turn Sheet directly onto the Fitted Sheet.

Visit our website at www.pm-med.de or watch our comprehensive product videos on our YouTube channel.

3.7 Approved accessories, combination with other products

Fitted Sheets are used in combination with the following products:

- PM-1582/2098/2099 Turn Sheets
- PM-7010/7011 Non-Slip Mats
- PM-8010 Bed Wedges

Petermann transfer aids are part of a comprehensive aid product range, which are effective and back-friendly for the user and comfortable and supportive for the patients. Additional Petermann products may be used according to the situation and clinical picture. Ask our experts. We are happy to help you!

3.7.1 Sets

| Item No. | Ingredients set |
|----------|------------------------|
| PM-1580 | PM-1582 Turn Sheet |
| | PM-1584 Fitted Sheet |
| | PM-7010 Non Slip Mat |
| PM-1685 | PM-2098.2 Turn Sheet |
| | PM-2091 Fixing Stripes |
| | PM-1584 Fitted Sheet |
| | PM-7010 Non Slip Mat |
| PM-1695 | PM-2098.2 Turn Sheet |
| | PM-2091 Fixing Stripes |
| | PM-1584 Fitted Sheet |
| | PM-8010 Bed Wedges |
| | PM-7010 Non Slip Mat |

| Item No. | Ingredients set |
|----------|-------------------------------|
| PM-1699 | PM-2099.1 Turn Sheet |
| | PM-2091 Fixing Stripes |
| | PM-1234 Air Pump |
| | PM-1233 Cover for Air Cushion |
| | PM-1232 Air Cushion |
| | PM-1584 Fitted Sheet |
| | PM-7010 Non Slip Mat |
| | PM-8010 Bed Wedges |

Please observe the user manuals of the other items as well.

4 Product marking / CE

The Petermann Fitted Sheet is a Class I Medical Device and meets the EU-Regulation 2017/745 of the European Parliament and Council dated 05.04.2017 for medical devices, also called MDR.

Petermann GmbH is certified in accordance with DIN EN ISO 13485 by TÜV Süd, Medical Devices - Quality management systems applicable to the design and development, production and distribution of aids for people of limited mobility.

5 Cleaning, care and disinfection

The product doesn't require any special maintenance. It can be machine washed at 90°C and dried at low temperatures (45° - 50°C). We recommend washing with mild detergents. The use of cleaning agents with bleaching ingredients or other chemical laundry additives, also as fabric softeners may lead to permanent deterioration of the product properties.

When necessary, we recommend wipe disinfection with the product Trionic® (DGHM/VAH approval, further information upon request). As alternatives to our proven disinfectant Trionic®, we can also recommend the following disinfectants for wipe disinfection:

- Mikrobac® Tissues*
- Mikrobac® forte*
- Bacillo® 30 Foam
- Bacillo® 30 Tissues

When using products marked with an *, wipe down again using a water-soaked cloth after disinfection has been completed, in order to remove residues and prevent build-up. For wipe disinfection, please be sure to always follow the instructions provided by the manufacturer.

Always follow the care instructions on the label of every product as well as the information in the respective user manual.

6 Technical Data

| Item no. | Material | Colour | Load-bearing capacity |
|----------|--------------------|--------|-----------------------|
| PM-1584 | Cotton, latex free | cream | 200 kg |

| Item no. | Description | Dimensions in mm | Weight | UDI |
|----------|--------------|------------------------|--------|-------------------------|
| PM-1584 | Fitted Sheet | 2000 x 900 x 200 mm | 1440 g | (01)04250355315847(10)x |

The manufacturing of our products is subject to highest quality standards. Due to usual manufacturing variations published dimensions and further information concerning the condition of the product are defined as target production specifications. Deviations which might occur are monitored by professional quality management procedures and do not represent a reduction in performance of the respective product.

7 Warranty

- The company Petermann warrants that the product is free from defects in material and workmanship.
- The warranty period is 3 years. **It does not cover defects attributable to abrasion, normal wear, improper use or improper cleaning.**
- The company Petermann is not liable for accidental or consequential injury or material damage.

8 Disposal information

There are no specific disposal regulations for Fitted Sheets, they can be disposed of in accordance with applicable national and local regulations.

Petermann GmbH
Waldweg 3
D-91601 Dombühl
Phone: +49 (0) 9868-9339-0
Email: petermann@pm-med.de

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