# Tragesitz Swing Transfer Seat Swing

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Bruksanvisning · Käsikirja

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### **1 Allgemeine Informationen**

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## Table of Contents

### **1 General Information**

- 1.1 Product versions
- 1.2 Accessories
- 1.3 Scope of delivery
- 1.4 Storage, transport
- 1.5 Service life / lifecycle

### **2 Safety information**

- 2.1 Symbols used
- 2.2 Function check

### **3 Using the product**

- 3.1 Indication and further use
- 3.2 Contraindications
- 3.3 Application risks
- 3.4 Areas of application
- 3.5 Application illustration
- 3.6 Application
- 3.7 Approved accessories, combination with other products

### **4 Product marking / CE**

### **5 Cleaning, care and disinfection**

### **6 Technical Data**

### **7 Warranty**

### **8 Disposal information**

# 1 General Information

Thank you for choosing a Petermann product.

**Please read the user manual carefully before using the product for the first time!**

The current understanding in modern occupational health and safety is that people in nursing professions experience significant work-related physical strain. Back injuries are common. There are many situations in which a patient has to be transferred from a bed, a wheeled stretcher, ambulance stretcher, an X-Ray table or a wheelchair to a different stretcher or seat. The strain on the user and the patient / care-dependent can be reduced significantly through the use of transfer aids.

The use of transfer aids should be conditional upon the user having learned to use the equipment properly. The user has to decide for themselves what equipment is suitable to the patient's situation and the intended sequence of movements.

The Transfer Seat Swing is part of a product line which is used specifically to support the transfer of patients in home care, hospitals and other health care facilities.

The product may only be used for the specified purpose and under the specified conditions.

The information in this user manual is for the patient who is being transferred and moved, as well as the user performing the transfer / position change.

Contact us if you require additional information or assistance using the Petermann product.

## 1.1 Product versions

**Item no.:** PM-4035 Transfer Seat Swing

## 1.2 Accessories

No accessories are available for this device.

## 1.3 Scope of delivery

Transfer Seat Swing  
User manual

## 1.4 Storage, transport

Store/transport                      at -5 C – 40 C  
Other:                                      protect from sunlight

## 1.5 Service life / lifecycle

If the product is used in accordance with the safety instructions, then the expected product service life is **five years**. However, the effective service life may vary depending on the frequency and intensity of use and the number of cleaning and disinfection cycles.

Perform regular function checks on the product and be sure to observe the safety information (see item 2).

# 2 Safety information

**WARNING indicates a potential danger which could lead to personal injury or material damage.**

- It is the responsibility of the organisation / the user to determine, on the basis of the circumstances (weight / health condition of the patient) and the capabilities of the user, whether several users may be required for the transfer and/or whether additional safety measures are required.
- The Transfer Seat Swing may only be used with the safety straps closed.
- Never leave the patient unattended during the transfer process.
- Observe the load handling regulations when performing patient lifting procedures.
- Before using the Transfer Seat Swing, ensure that the patient is not suffering from any injuries or weakness which would preclude use of the device.
- Pull the straps so tight that the straps lie firmly against the patient but don't cut in, and be sure to release the straps after every transfer.
- Defective products may no longer be used.
- Ensure that the transfers are performed by the users in a back-friendly manner.
- Weight limits must be observed (200 kg)
- Transfer products are subject to a natural ageing process. Inspect the equipment regularly to ensure proper functionality
- The use of non-standard cleaning agents or bleaching agents, alcohols, fabric softeners or other chemical laundry additives may lead to permanent deterioration of the sliding mat properties.

## 2.1 Symbols used

	Wipe off		max. load capacity see item 6
	Do not bleach		Manufacturer
	Do not tumble		Batch no. / lot no.
	Do not iron		Identification code
	Do not dry clean		Conformity mark
	Warning		Item number
	Observe the user manual		Medical device label

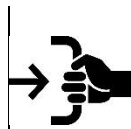
## 2.2 Function check

Transfer products are also subject to a natural ageing process. Inspect the equipment regularly and carefully, in particular before each use.

### Visual inspection

Ensure that material, seams, handle loops and buckles are undamaged and in functional order.

### Load test handles

Pull firmly on the handle loops - Check material and seams

### Load test buckles

Pull firmly on the closed buckle - Check buckle and seams

➤ **Defective products may no longer be used!**

## 3 Using the product

### 3.1 Indication and further use

Due to the wide range of possible applications, the indications cannot be tied to a specific clinical picture, since the Transfer Seat Swing is suitable for all applications involving people who require assistance during transfers. The product can be used as a support for the following diagnoses:

- Carrying aid in order to safely carry nonambulatory patients even out of difficult to access situations
- Carrying aid also for evacuation if only the staircase is available for the rescue

#### **This product may be reused.**

Before reusing the product, be sure to perform a thorough function check (see item 2.2) as well as cleaning and disinfection of the product. (see item 5)

## 3.2 Contraindications

Contraindications cannot be tied to a specific clinical picture. Observe the safety information under item 2.

- For pathologically agitated patients

## 3.3 Application risks

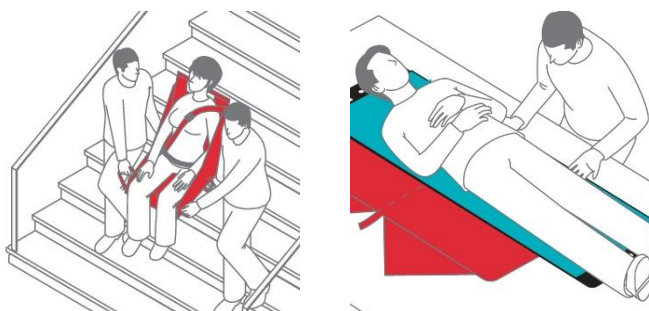
- The patient is assessed incorrectly → risk of falling
- Too few people are carrying → risk of falling
- Lack of stability → risk of falling

## 3.4 Areas of application

The product can be used in home care and other health care facilities as well as hospitals.

## 3.5 Application illustration

## 3.6 Application

For stairs and transfers, the Transfer Seat Swing facilitates a largely controlled transfer of people especially in situations where patient-assisted aids cannot be used due to the given conditions.

### 3.6.1 Preparation

- Release both safety catches on the outer side sections
- Unfold the Transfer Seat Swing to form a level surface to lie on
- Consider the weight and condition of the patient
- Consider the difficulty of the transfer situation
- Coordinate the sequence of movements with the patients and assistants

### 3.6.2 Placement

- Place the person on their side
- Fold the side pieces which will face down under the middle of the Transfer Seat Swing.
- Place the Transfer Seat Swing against the patient's back → Seat at buttock height
- Move the safety belts to the outside
- Place the patient on their back
- Raise the patient
- Close both safety catches on the outer side sections
- Close and tighten the safety belts

### 3.6.3 Transfer

- Keep the Transfer Seat Swing close to your body to minimize leverage and to reduce back strain.

Visit our website at [www.pm-med.de](http://www.pm-med.de) or watch our comprehensive product videos on our YouTube channel.

## 3.7 Approved accessories, combination with other products

Petermann transfer aids are part of a comprehensive aid product range, which are effective and back-friendly for the user and comfortable and supportive for the patients. Additional Petermann products may be used according to the situation and clinical picture. Ask our experts. We are happy to help you!

## 4 Product marking / CE

The Petermann Transfer Seat Swing is a Class I Medical Device and meets the EU-Regulation 2017/745 of the European Parliament and Council dated 05.04.2017 for medical devices, also called MDR.

Petermann GmbH is certified in accordance with DIN EN ISO 13485 by TÜV Süd, Medical Devices - Quality management systems applicable to the design and development, production and distribution of aids for people of limited mobility.

## 5 Cleaning, care and disinfection

<b>Cleaning</b>	<b>Disinfection (if necessary)</b>
Damp cleaning	Recommended disinfectants for wipe disinfection: <ul style="list-style-type: none"><li>- Trionic® (DGHM/VAH approval, further information on request)</li><li>- Mikrobac® Tissues*</li><li>- Mikrobac® forte*</li><li>- Bacillo® 30 Foam</li><li>- Bacillo® 30 Tissues</li><li>- Bacillo® 30 Sensitive Tissues</li><li>- Mikrobac® Virucidal Tissues*</li></ul>
Warm water / mild detergent	
Do not use abrasive or bleaching agents!	

Do not use sharp-edged objects!

\* wipe with a cloth soaked in water after disinfection to avoid residues or layer build-up. Please always follow the manufacturer's manuals for use when wiping disinfection.

Be sure to follow the care instructions on the label of each product as well as the manuals for use.

## 6 Technical Data

Item no.	Material	Colour	Load-bearing capacity
PM-4035	Polyamide, latexfrei	red	180 kg

Item no.	Description	Dimensions in mm	Weight	UDI
PM-4035	Transfer Seat Swing	380 x 430 x 550	2300	(01)04250355340351(10)x

The manufacturing of our products is subject to highest quality standards. Due to usual manufacturing variations published dimensions and further information concerning the condition of the product are defined as target production specifications. Deviations which might occur are monitored by professional quality management procedures and do not represent a reduction in performance of the respective product.

## 7 Warranty

- The company Petermann warrants that the product is free from defects in material and workmanship
- The warranty period is 3 years. **It does not cover defects attributable to abrasion, normal wear, improper use or improper cleaning**
- The company Petermann is not liable for accidental or consequential injury or material damage

## 8 Disposal information

There are no specific disposal regulations for Transfer Seat Swing, it can be disposed of in accordance with applicable national and local regulations.

**Petermann GmbH**  
**Waldweg 3**  
**D-91601 Dombühl**  
**Phone: +49 (0) 9868-9339-0**  
**Email: petermann@pm-med.de**

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