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Table of Contents

1 General Information

- 1.1 Product versions
- 1.2 Accessories
- 1.3 Scope of delivery
- 1.4 Storage, transport
- 1.5 Service life / lifecycle

2 Safety information

- 2.1 Symbols used
- 2.2 Function check

3 Using the product

- 3.1 Indication and further use
- 3.2 Contraindications
- 3.3 Application risks
- 3.4 Areas of application
- 3.5 Application illustration
- 3.6 Application
- 3.7 Approved accessories, combination with other products

4 Product marking / CE

5 Cleaning, care and disinfection

6 Technical Data

7 Warranty

8 Disposal information

1 General Information

Thank you for choosing a Petermann product.

Please read the user manual carefully before using the product for the first time!

Bed Wedges are part of a product line which is used specifically to assist with positioning of patients in hospitals and other health care facilities in the field of emergency care, diagnostics, nursing and therapy, as well as in home care.

The product may only be used for the specified purpose and under the specified conditions.

The information in this user manual is for the patient who is being transferred and moved, as well as the user performing the transfer / position change.

Contact us if you require additional information or assistance using the Petermann product.

1.1 Product versions

Item no.: PM-8010 Bed Wedges set includes:

- 2 x PM-8012
- 1 x PM-8015

1.2 Accessories

No accessories are available for this device.

1.3 Scope of delivery

Bed Wedges set includes:

2 x Bed Wedges 250 x 220 x 130/0 mm

1 x Bed Wedge 500 x 220 x 130/0 mm

User manual

1.4 Storage, transport

Store/transport at -5 C – 40°C
Other: protect from sunlight

1.5 Service life / lifecycle

If the product is used in accordance with the safety instructions, then the expected product service life is **three years**. However, the effective service life may vary depending on the frequency and intensity of use and the number of cleaning and disinfection cycles.

Perform regular function checks on the product and be sure to observe the safety information (see item 2).

2 Safety information

WARNING indicates a potential danger which could lead to personal injury or material damage.

- It is the responsibility of the organisation / the user to determine, on the basis of the circumstances (weight / health condition of the patient) and the capabilities of the user, whether several users may be required for the transfer and/or whether additional safety measures are required.
- Defective products may no longer be used.
- Any use of the Bed Wedges other than the intended use is prohibited!
- Ensure that the positionings are performed by the users in a back-friendly manner.
- In case of direct skin contact, observation of changes in skin appearance is indicated.
- Don't use any pointy objects in combination with the non-slip cushion, since this could result in damage to the product.
- The use of disinfectants containing alcohol should be avoided. Coloured disinfectants (containing iodine) may cause slight discolouration.

2.1 Symbols used

	Wipe off		Observe the user manual
	Do not bleach		Manufacturer
	Do not tumble		Batch no. / lot no.
	Do not iron		Identification code
	Do not dry clean		Conformity mark
	Warning		Item number
	Medical device label		

2.2 Function check

Transfer products are also subject to a natural ageing process. Inspect the equipment regularly and carefully, in particular before each use.

Visual inspection

Ensure that material, seams, handle loops and buckles are undamaged and in functional order.

➤ **Defective products may no longer be used!**

3 Using the product

3.1 Indication and further use

Due to the wide range of possible applications, the indications cannot be tied to a specific clinical picture, since positioning aids are suitable for all applications involving people who require assistance. The product can be used in support of the following diagnoses:

- Support for positioning
- Stabilisation for positioning
- Pressure relief for positioning
- Further applications in combination with turn sheets / sliding mats.

This product may be reused.

Before reusing the product, be sure to perform a thorough function check (see item 2.2) as well as cleaning or disinfection of the product. (see item 5)

3.2 Contraindications

Contraindications cannot be tied to a specific clinical picture. Under consideration of the safety information (see item 2), the product may not be used if the patient has any of the following conditions:

- Individually perceived pain

3.3 Application risks

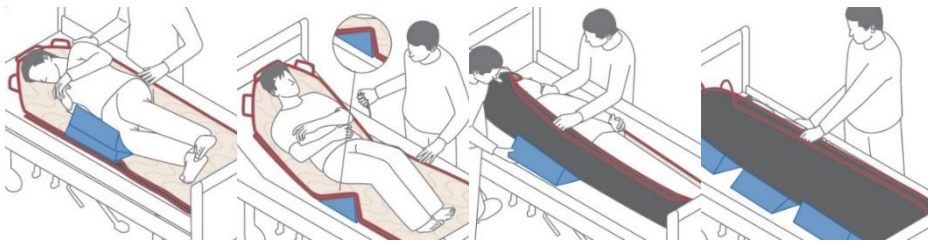
- Increasing the support pressure on wounds or bedsores. → worsening of the skin condition
- Insufficient stabilisation → positioning is not retained
- Improper placement → positioning is not retained

Please contact us in these cases, so that we can offer you an alternate solution.

3.4 Areas of application

The product can be used in home care and other health care facilities as well as hospitals in the field of emergency care, diagnostics, nursing and therapy.

3.5 Application illustration

3.6 Application

The Bed Wedges were developed for an activating and supporting positioning of patients. Their versatility allows for goal-oriented and purposeful positioning in accordance with the planned care, therapeutic or diagnostic measures. The use of Bed Wedges has proven particularly effective at night, in order to ensure that the patient's sleep is as uninterrupted as possible.

Even with the best possible positioning, the patient should be repositioned and/or the positioning should be checked regularly in accordance with the risk level.

Application examples

- Safe and stable positioning up to 135° → Dressing changes on the rump, basic care and change of incontinence material.
- Securing a sitting position in bed → Food intake
- Lateral positioning
- Micro position changes

Visit our website at www.pm-med.de or watch our comprehensive product videos on our YouTube channel.

3.7 Approved accessories, combination with other products, sets

Bed Wedges are used in combination with the following products:

- Sliding Mats
- Turn Sheets
- PEM-Disposable Sliding Foil

Petermann transfer aids are part of a comprehensive aid product range, which are effective and back-friendly for the user and comfortable and supportive for the patients. Additional Petermann products may be used according to the situation and clinical picture. Ask our experts. We are happy to help you!

Sets

Item No.	Ingredients set
PM-1695	PM-2098.2 Turn Sheet
	PM-2091 Fixing Stripes
	PM-1584 Fitted Sheet
	PM-7010 Non Slip Mat
	PM-8010 Bed Wedges
PM-1699	PM-2099.1 Turn Sheet Air Sheet
	PM-2091 Fixing Stripes
	PM-1234 Air Pump
	PM-1233 Cover for Air Cushion
	PM-1232 Air Cushion
	PM-1584 Fitted Sheet
	PM-7010 Non Slip Mat
	PM-8010 Bed Wedges

Please observe the user manuals of the other items as well.

4 Product marking / CE

The Petermann Bed Wedges are a Class I Medical Device and meets the EU-Regulation 2017/745 of the European Parliament and Council dated 05.04.2017 for medical devices, also called MDR.

Petermann GmbH is certified in accordance with DIN EN ISO 13485 by TÜV Süd, Medical Devices - Quality management systems applicable to the design and development, production and distribution of aids for people of limited mobility.

5 Cleaning, care and disinfection

Cleaning	Desinfection (if necessary)
Damp cleaning	Recommended disinfectants for wipe disinfection: <ul style="list-style-type: none"> - Trionic® (DGHM/VAH approval, further information on request) - Mikrobac® Tissues* - Mikrobac® forte* - BacilloI® 30 Foam - BacilloI® 30 Tissues - BacilloI® 30 Sensitive Tissues - Mikrobac® Virucidal Tissues*
Warm water / mild detergent	
Do not use abrasive or bleaching agents!	

Do not use sharp-edged objects!

* wipe with a cloth soaked in water after disinfection to avoid residues or layer build-up. Please always follow the manufacturer's manuals for use when wiping disinfection.

Be sure to follow the care instructions on the label of each product as well as the manuals for use.

6 Technical Data

Item no.	Material	Colour	Load-bearing capacity
PM-8010	PU-anti slip	blue	N/A

Item no.	Description	Dimensions in mm	Weight	UDI
PM-8010	Bed Wedges set includes:		500 g	(01)04250355380104(10)x
PM-8012	2 x Bed Wedges	250 x 220 x 130/0	125 g	(01)04250355380128(10)x
PM-8015	1 x Bed Wedge	500 x 220 x 130/0	250 g	(01)04250355380159(10)x

The manufacturing of our products is subject to highest quality standards. Due to usual manufacturing variations published dimensions and further information concerning the condition of the product are defined as target production specifications. Deviations which might occur are monitored by professional quality management procedures and do not represent a reduction in performance of the respective product.

7 Warranty

- The company Petermann warrants that the product is free from defects in material and workmanship.
- The warranty period is 1 years. **It does not cover defects attributable to abrasion, normal wear, improper use or improper cleaning.**
- The company Petermann is not liable for accidental or consequential injury or material damage.

8 Disposal information

There are no specific disposal regulations for Bed Wedges, they can be disposed of in accordance with applicable national and local regulations.

Petermann GmbH
Waldweg 3
D-91601 Dombühl
Phone: +49 (0) 9868-9339-0
Email: petermann@pm-med.de

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