Non Slip Mats

Manual

Bruksanvisning

Table of Contents

1 General Information

- 1.1 Product versions
- 1.2 Accessories
- 1.3 Scope of delivery
- 1.4 Storage, transport
- 1.5 Service life / lifecycle

2 Safety information

- 2.1 Symbols used
- 2.2 Function check

3 Using the product

- 3.1 Indication and further use
- 3.2 Contraindications
- 3.3 Application risks
- 3.4 Areas of application
- 3.5 Application illustration
- 3.6 Application
- 3.7 Approved accessories, combination with other products

4 Product marking / CE

5 Cleaning, care and disinfection

6 Technical Data

7 Warranty

8 Disposal information

1 General Information

Thank you for choosing a Petermann product.

Please read the user manual carefully before using the product for the first time!

The current understanding in modern occupational health and safety is that people in nursing professions experience significant work-related physical strain. Back injuries are common. There are many situations in which a patient has to be transferred from a bed, a wheeled stretcher, ambulance stretcher, an X-Ray table or a wheelchair to a different stretcher or seat. The strain on the user and the patient / care-dependent can be reduced significantly through the use of raising aids.

The use of transfer aids should be conditional upon the user having learned to use the equipment properly. The user has to decide for themselves what equipment is suitable to the patient's situation and the intended sequence of movements.

The Non Slip Mat is part of a product line which is used specifically to assist with positioning of patients in hospitals and other health care facilities, as well as in home care.

The product may only be used for the specified purposes and conditions.

The information in this user manual is for the patient as well as for the use.

Contact us if you require additional information or assistance using the Petermann product.

1.1 Product versions

Item no.: **PM-7010** Non Slip Mat
PM-7011 Non Slip Mat
PM-7007 Non Slip Mat on Roll

1.2 Accessories

No accessories are available for this device.

1.3 Scope of delivery

Non Slip Mat
User manual

1.4 Storage, transport

Store/transport at -5°C – 40°C
Other: protect from sunlight

1.5 Service life / lifecycle

If the product is used in accordance with the safety instructions, then the expected product service life is **two years**. However, the effective service life may vary depending on the frequency and intensity of use and the number of cleaning and disinfection cycles.

Perform regular function checks on the product and be sure to observe the safety information (see item 2).

2 Safety information

WARNING indicates a potential danger which could lead to personal injury or material damage.

- It is the responsibility of the organisation / the user to determine, on the basis of the circumstances (weight / health condition of the patient) and the capabilities of the user, whether several users may be required for the transfer and/or whether additional safety measures are required.
- Defective products may no longer be used.
- Any use of the Non Slip Mat other than the intended use is prohibited!
- Ensure that the transfers are performed by the users in a back-friendly manner.
- Don't use any pointy objects in combination with the Non Slip Mat, since this could result in damage to the product.
- Positioning aids are subject to a natural ageing process. Inspect the equipment regularly to ensure proper functionality.
- The use of non-standard cleaning agents or bleaching agents, alcohols, fabric softeners or other chemical laundry additives may lead to permanent deterioration of the Non Slip Mat properties.

2.1 Symbols used

	Wipe off		max. load capacity see item 6
	Do not bleach		Manufacturer
	Do not tumble		Batch no. / lot no.
	Do not iron		Identification code
	Do not dry clean		Conformity mark
	Warning		Item number
	Observe the user manual		Medical device label

2.2 Function check

Positioning aids are also subject to a natural ageing process. Inspect the equipment regularly and carefully, in particular before each use. Ensure that the product is in sound condition and that the material is not damaged. The rough surface of the anti-slip material prevents slipping. Check these properties - ideally in comparison to a new product.

➤ **Defective products may no longer be used!**

1 Using the product

1.1 Indication and further use

Due to the wide range of possible applications, the indications cannot be tied to a specific clinical picture, since positioning are suitable for all applications involving people requiring support in good seated positions. The Non Slip Mat reduces extension at the hip level and thus has a physiological anti-sliding effect.

This product may be reused.

Before reusing the product, be sure to perform a thorough function check (see item 2.2) as well as cleaning or disinfection of the product (see item 5).

1.2 Contraindications

Contraindications cannot be tied to a specific clinical picture. Under consideration of the safety information (see item 2), the product may not be used if the patient has any of the following conditions:

- Diagnoses for which "sitting" isn't possible
- Fractures in the pelvic region
- Pain in the pelvic region


1.3 Application risks

- When placing the Non Slip Mat, the specifics of the clinical picture must be determined and taken into consideration → risk of worsening of the clinical picture
- The patient slides down despite positioning aid → positioning is not retained

1.4 Areas of application

The product can be used in home care and other health care facilities as well as hospitals.

1.5 Application illustration

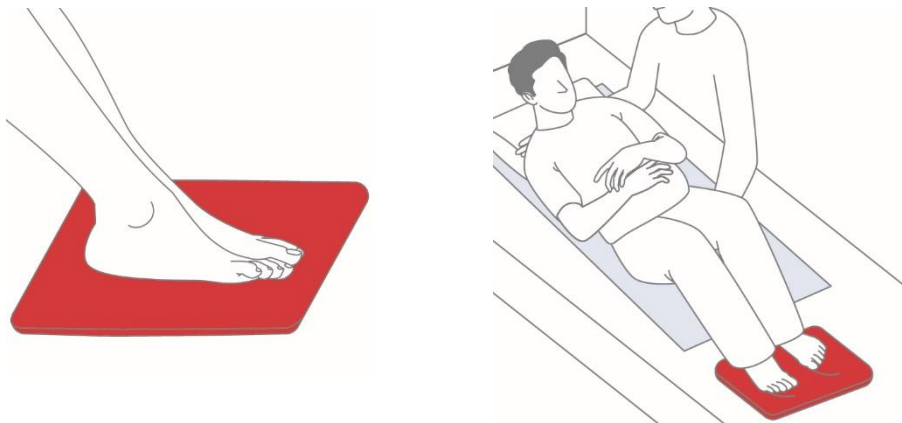1.6 Application

The Alpha Non Slip Mat prevents the feet from slipping in bed. When placed underneath the client's feet in bed, the non-slip mat helps to prevent slipping while repositioning or raising the patient. It also makes it possible for the client moving toward the top of the bed by pressing with their feet.

The Alpha Non-Slip Mat on Role is a non-slip plastic film with a high level of friction that comes on a role. Cut the section or shape you want, and let the product cater to several anti-slip needs.

Visit our website at www.pm-med.de or watch our comprehensive product videos on our YouTube channel.

1.7 Approved accessories, combination with other products

Petermann Non Slip Mat is part of a comprehensive aid product range, which are effective and back-friendly for the user and comfortable and supportive for the patients. Additional Petermann products may be used according to the situation and clinical picture. Ask our experts. We are happy to help you!

2 Product marking / CE

Petermann GmbH is certified in accordance with DIN EN ISO 13485 by TÜV Süd, Medical Devices - Quality management systems applicable to the design and development, production and distribution of aids for people of limited mobility.

3 Cleaning, care and disinfection

All Petermann repositioning and transfer aids can be wet cleaned. Use warm water and a mild cleaning agent. If applicable, wipe the product off with a moist cloth and then let it dry thoroughly. Don't use any scouring or bleaching agents or any sharp-edged objects for cleaning.

When necessary, we recommend wipe disinfection with the product Trionic® (DGHM/VAH approval, further information upon request). As alternatives to our proven disinfectant Trionic®, we can also recommend the following disinfectants for wipe disinfection:

- Mikrobac® Tissues*
- Mikrobac® forte*
- Bacillol® 30 Foam
- Bacillol® 30 Tissues

When using products marked with an *, wipe down again using a water-soaked cloth after disinfection has been completed, in order to remove residues and prevent build-up. For wipe disinfection, please be sure to always follow the instructions provided by the manufacturer.

Always follow the care instructions on the label of every product as well as the information in the respective user manual.

4 Technical Data

Item no.	Material	Colour
PM-7010	Polymeric Compound	red
PM-7011	Polymeric Compound	red
PM-7007	Polymeric Compound	blue

Item no.	Description	Dimensions in mm/inches
PM-7010	Alpha Non Slip Mat Small	35 x 25 cm / 13,7" x 9,8"
PM-7011	Alpha Non Slip Mat Medium	45 x 38 cm / 17,7" x 14,9"
PM-7007	Alpha Non Slip Mat on Roll	40 x 200 cm / 15,7" x 78,7"

The manufacturing of our products is subject to highest quality standards. Due to usual manufacturing variations published dimensions and further information concerning the condition of the product are defined as target production specifications. Deviations which might occur are monitored by professional quality management procedures and do not represent a reduction in performance of the respective product.

5 Warranty

- The company Petermann warrants that the product is free from defects in material and workmanship.
- The warranty period is 1 year. **It does not cover defects attributable to abrasion, normal wear, improper use or improper cleaning.**
- The company Petermann is not liable for accidental or consequential injury or material damage.

6 Disposal information

There are no specific disposal regulations for Non Slip Mats, they can be disposed of in accordance with applicable national and local regulations.

Petermann GmbH
Waldweg 3
D-91601 Dombühl
Phone: +49 (0) 9868-9339-0
Email: petermann@pm-med.de

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