

# Low Basic Sling



Supportive  
Two fabrics  
Easy

## Low Basic Sling

Our Low Basic Sling is a good solution for users who have torso stability and upper body control. The design is similar with the Basic Sling, however it has a shorter back support. The Low Basic Sling provides support from the tailbone up under the arms and under the legs. The arms can be kept inside, but are mostly kept outside of the sling. It is a perfect sling for a more active user. The Low Basic Sling is designed for two-, three- and four-point suspension.



### Two fabrics

The Low Basic Sling is a comfortable sling that is available in two fabric choices - Polyester and Polyester Net. The Polyester Net materials allow moisture and heat to transfer efficiently from the client which allows the sling to be left behind the client for a short period of time after the transfer is complete. The Polyester Net fabric is flexible, airy and dries quickly which makes it suitable for use in bathing environments.

### Upright Seating

Our Low Basic Sling offers support so that the client can have their arms on the outside of the sling. They are suitable for patients with trunk stability and upright seating which is especially helpful to

ensure achieving proper positioning when the patient is placed in a chair or other seating. They can be used with a 2-point, 3-point or 4-point suspension depending on the need.

### Easy Care and Easy Fit

Our slings are manufactured to the highest quality standards. They feature reinforced straps and high strength nylon webbing around the sling to make them strong enough to resist even the most active use. Each sling label is colour coded to ensure quick and easy selection of the correct sling and the straps are colour coded for easy determination of required positioning.



Walking Aids



Healthcare Beds



Lifting Solutions



Hygiene



Convertible Chairs

## Accessories



### Extension Loops

Used to extend lift straps in order to alter lifting position or to hold leg parts together. Assorted lengths. Part no.:21180-\*



### Leg Padding and Supports

Ideal for users with sensitive skin or pain. Part no.: 21146-S, M, L Reinforced supports. One size. Part no. 25102



### Upholstered Cover

Protects user from hanger bar impact. Not for use in a wet environment. Part no. 50866



### Hanger Bars for 2-, 3- or 4-point Suspension

Part no. 57551, 55764 (2- or 3-Point - Quick Release), 55619 (4-Point - Attach to leg hook)

## Lifting Situation




### Use to/from a bed

Suitable for transfers and repositioning to and from a bed. Can be left behind patient for short period of time.




### Use to/from a chair

Suitable for transfers and repositioning to and from a chair. Can be left behind patient for short period of time.

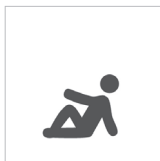

### Use to/from the floor

Suitable for transfers from the floor.

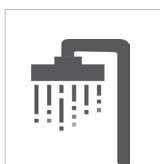

### Use for showering


Only Polyester Net fabric can be used for this application.

## Care and Features




### Polyester Net

Soft, thin material that breathes and dries quickly. Ideal for bathing or showering..

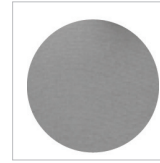

### Polyester



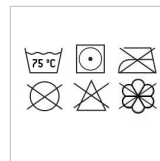



Durable material with a low coefficient of friction, making it easy to apply and clean. Equipped with handles.




### Care

Care instructions located on sling label. Use a hospital grade laundry bag while laundering

## Technical specifications

Article number		Size	Colour	A	B	C
Polyester net	Polyester					
25047-JS	25072-JS	Junior Small		22 cm / 8,7"	65 cm / 25,6"	30/11,8"
25047-JM	25072-JM	Junior Medium		28 cm / 11"	71 cm / 28"	37 cm / 14,6"
25047-S	25072-S	Small		35 cm / 13,8"	83 cm / 32,7"	42 cm / 16,5"
25047-M	25072-M	Medium		40 cm / 15,7"	94 cm / 37"	43 cm / 16,9"
25047-L	25072-L	Large		44 cm / 17,3"	104 cm / 40,9"	45 cm / 17,7"
25047-XL	25072-XL	Extra Large		48 cm / 18,9"	108 cm / 42,5"	48 cm / 18,9"
25047-XXL	25072-XXL	Extra Extra Large		52 cm / 20,5"	111 cm / 43,7"	52 cm / 20,5"
Maximum user weight: 300 kg / 660 lbs						

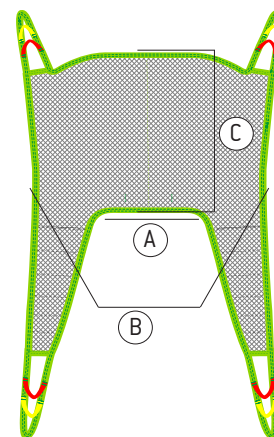

\* Available in assorted lengths: 10cm/3.9" [21180-10], 15cm/5.9" [21180-15], 20cm/7.8" [21180-20], 25cm/9.8" [21180-25], 35cm/13.7" [21180-35]



*We improve freedom of movement  
and quality of care for people.*

[www.humancaregroup.com](http://www.humancaregroup.com)



### HUMAN CARE SWEDEN (HQ)

Årstaängsvägen 21B  
117 43 Stockholm  
+46 8 665 35 00  
info@humancaregroup.com

### HUMAN CARE CANADA

10-155 Colonnade Road  
ON K2E 7K1 Ottawa  
+1 613 723 6734  
info.ca@humancaregroup.com

### HUMAN CARE THE NETHERLANDS

Elsperweg 124  
8076 PA Vierhouten  
+31 577 41 21 71  
info.nl@humancaregroup.com

### HUMAN CARE UNITED STATES

8006 Cameron rd, Suite K  
TX 78754 Austin  
+1 512 76 7199  
info.us@humancaregroup.com