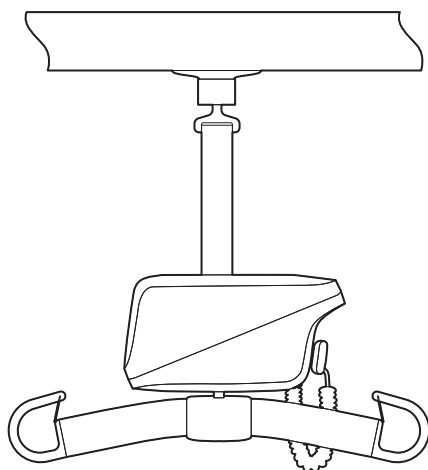
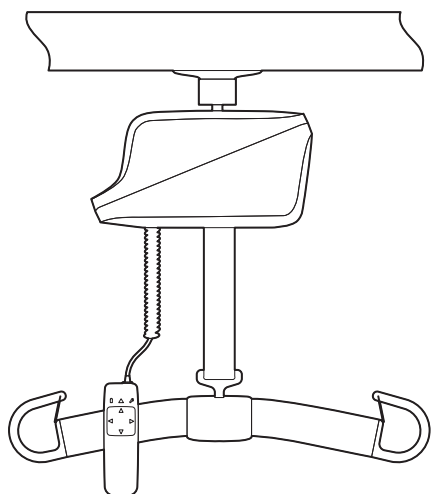
HUMAN CARE

Your Life. Your Way.

Cosmos

Service Manual / Servicemanual /
Servicehandbuch / Servicehandleiding

Art. Nr: 56100/56110/56120

Content

English:

Cosmos:

EXPLODED VIEW	3
SPARE PART KITS	4
ASSEMBLY INSTRUCTIONS: Removing the cover	6
ASSEMBLY INSTRUCTIONS: Replacing the cover	7
ASSEMBLY INSTRUCTIONS: Replacing the battery	8
ASSEMBLY INSTRUCTIONS: Removing the lift belt	9
ASSEMBLY INSTRUCTIONS: Replacing the lift belt	10
ASSEMBLY INSTRUCTIONS: Removing the hand control	14
ASSEMBLY INSTRUCTIONS: Replacing the hand control	15
ANNUAL INSPECTION COSMOS	16

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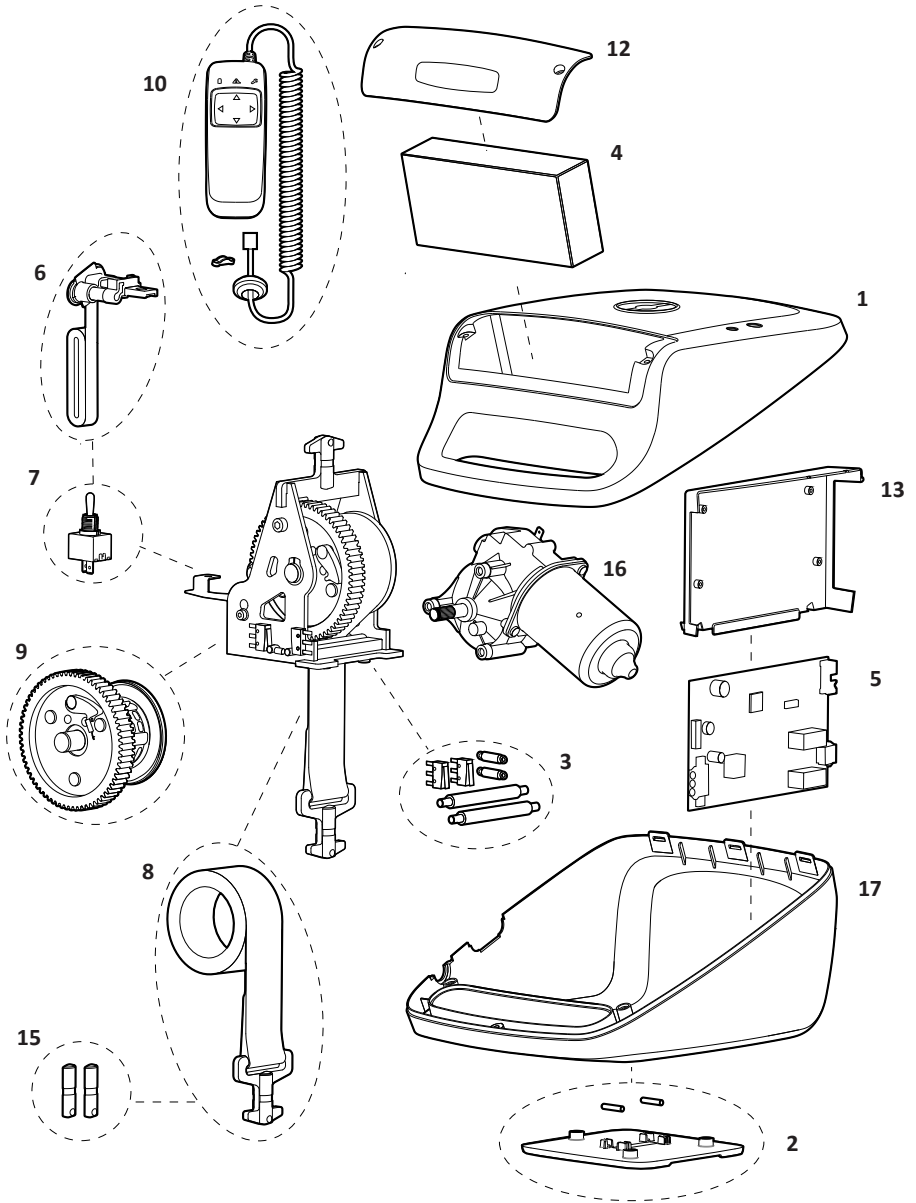
All spare parts for the ceiling lift are sold as kits. Circled and pointed parts shows the kits that are available. Detailed list with included parts is shown at the next page.

Kit number in exploded view	Part no.	Spare parts kit	Included in kit	Quantity of pieces in kit
1	90569	Top cover Cosmos-kit	Screw PT WN1452 K30x40 FZB	3
			Top cover Cosmos	1
2	90586	Platform Cosmos-kit	Platform lift	1
			Pins	2
3	90572	Tension sensor/belt stop Cosmos-kit	Tension sensor assembly	1
			End stop sensor assembly	1
			Screw MC6S M2x10 A2	4
			End stop shaft	2
			Tension tube	2
			Spring	2
4	90573	Battery ceiling lift Cosmos-kit	Lithium-Ion battery pack, complete (25.2V DC 4.8Ah)	1
5	90576	Main board Cosmos-kit	Main board Cosmos	1
			Screw PT WN5412 30x6 FZB	4
6	90579	Safety Stop belt Cosmos-kit	Safety Stop belt	1
			Outlet safety	1
7	93612	Emergency stop and battery cable Cosmos-kit	Safety switch assembly	1
8	90580	Lift belt Cosmos-kit	Lift Belt Cosmos	1
9	94324	Belt roll Cosmos-kit	Philips screw MC6S M6x16 A2	6
			Belt Roll assembly	1
			Shaft Belt Roll	1
			C-clip	1
			Spacer shaft roller	1
10	90574	Hand control Cosmos-kit	Hand control Cosmos, including cable	1
			Strain Relief Clamp cc12	1
11 - not in view	94318	Charger kit EU	Battery Charger	1
			EU plug	1
11 - not in view	94319	Charger kit UK	Battery Charger	1
			UK plug	1
11 - not in view	94320	Charger kit AU	Battery Charger	1
			AU plug	1

Kit number in exploded view	Part no.	Spare parts kit	Included in kit	Quantity of pieces in kit
11 - not in view	94321	Charger kit NA	Battery Charger	1
			NA plug	1
11 - not in view	94322	Charger kit JP	Battery Charger	1
			JP plug	1
12	90581	Cover battery lid Cosmos-kit	Cover battery lid Cosmos, complete	1
			Screw MC6S M4x10 A2	2
13	90582	Cover Mainboard Cosmos-kit	Cover main board Cosmos	1
			Screw PTWN 5412 40x14 FZB	2
14 - not in view	90584	Screws Platform Cosmos-kit	Screw PTWN 5412H 40x14 FZB	3
15	90585	Interface steering	Interface steering	2
16	54075	Motor	Motor Cosmos	1
17	90583	Cover Bottom Main Unit	Bottom Cover Cosmos	1
			Screw PT WN1452 K30x40 FZB	3

ASSEMBLY INSTRUCTIONS

Tools Required

Screw driver

Allen key 3 mm

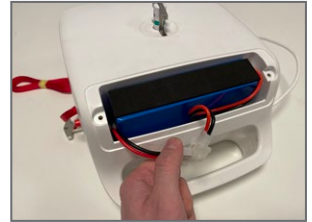
Quick Tip! Remove the hanger bar for easier handling

1. Activate the emergency stop by pulling the red ribbon.

2. Remove the battery cover with the Allen key.

3. Disconnect the battery.

4. Remove the battery.

5. Remove screws from the bottom cover.

6. Remove the top cover.

7.1. Remove the top cover (take note of the cables and the connectors)

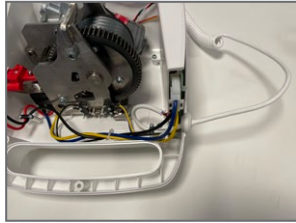
7.2. Remove the top cover (take note of the cables and the connectors)

ASSEMBLY INSTRUCTIONS

1. Insert the connector into the top cover.

2. Double check that all cables are correctly connected. Connect the battery and test the functions before assembly.

3. Place the top cover on lift.

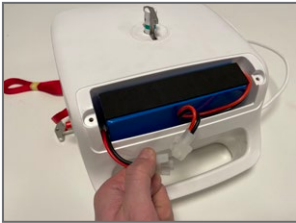
3.5. Assemble the top cover in its correct place.


4. Ensure the battery cable is pulled up and placed in the correct position.

5. Screw on the top cover with the screw driver.

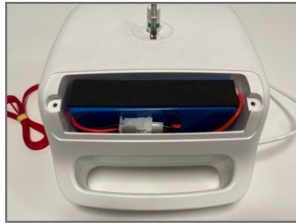
6. Place the battery.

7. Connect the cables to the battery.

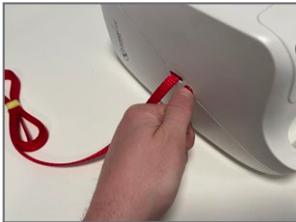8. Screw on the battery cover with the allen key.

9. Deactivate the emergency stop by pushing the red button.

Replacing the battery

ENG

ASSEMBLY INSTRUCTIONS

Tools Required

Screw driver

Allen key 3 mm

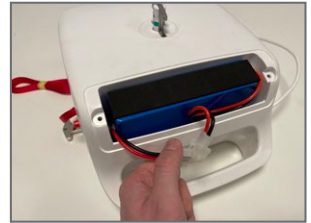
Quick Tip! Remove the hanger bar for easier handling

1. Activate the emergency stop by pulling the red ribbon.

2. Remove the battery cover with Allen Key.


3. Disconnect the battery.


4. Remove the battery.

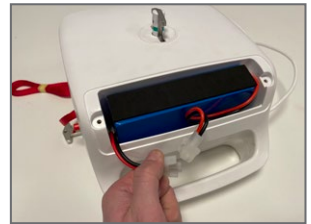5. Take the new battery.

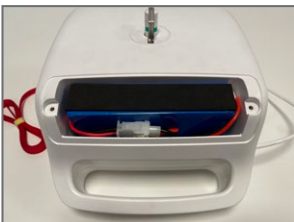6. Place and connect the new battery.

7. This is what the battery and its cables should look like.

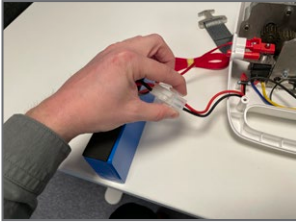
8. Mount the battery cover and screw on.


9. Deactivate the emergency stop by pushing the red button.


ASSEMBLY INSTRUCTIONS

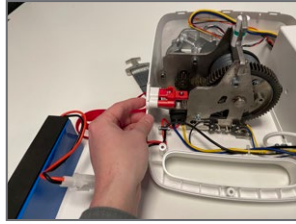
1. Remove the top cover. Connect the battery.

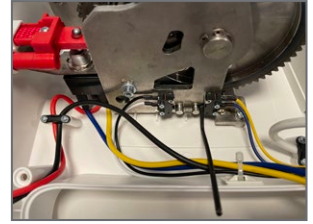
2. Deactivate the emergency stop by pushing the red button.

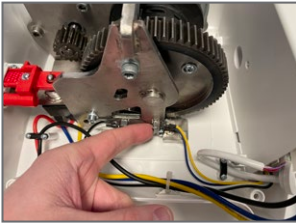3.1. Activate the sensor with the Allen Key 3 mm (or see step 3.2.)

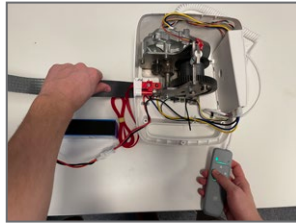
3.2. Activate the sensor with a finger.

4. Extend the belt all the way out by pushing the button on the hand control.

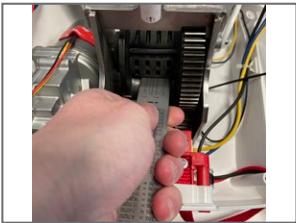
5. Activate the emergency stop by pulling the red ribbon.

6. Push the belt out.

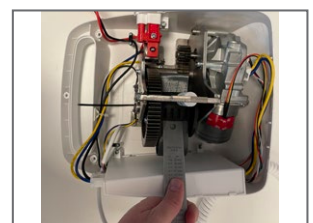
7. Create a loop.

8. Push out the end of the belt.

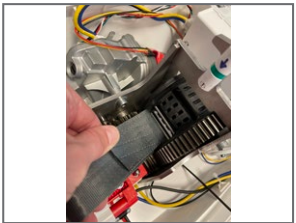9. Remove the pin.

10. Pull out belt from the belt roll.

11. Pull out the belt from the main unit.

ASSEMBLY INSTRUCTIONS

1. Put in the new belt.

Important Information:

Start to push the belt through the slot in the Belt Roll.

Note: folding should not be visible when the handle of the plastic cover is to the right.

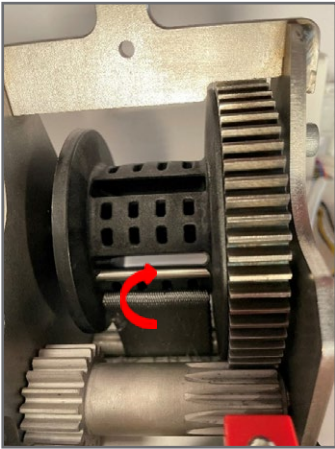
CORRECT

INCORRECT


ASSEMBLY INSTRUCTIONS


2. Put the belt in the first belt roll slot with the metal part.

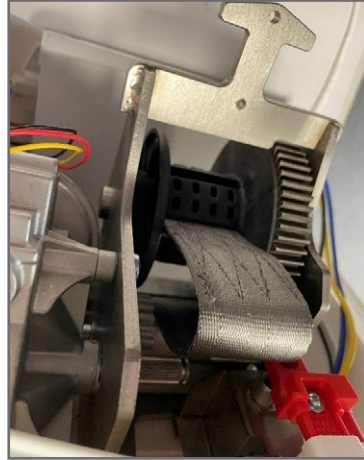
Important to start with the slot where the metal Belt Roll Pin is located.

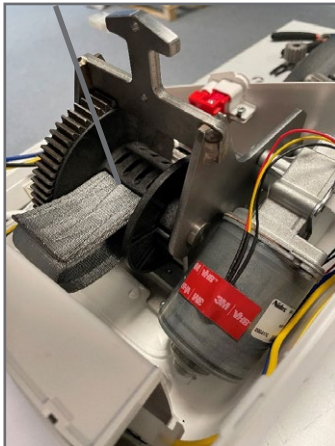
Quick Tip! Use a tool, like a spatula, to help guide the lift belt.

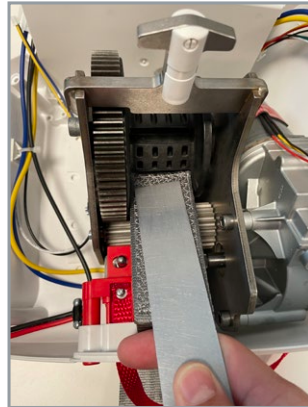
3. Put the belt back in through the other belt roll slot.

ASSEMBLY INSTRUCTIONS

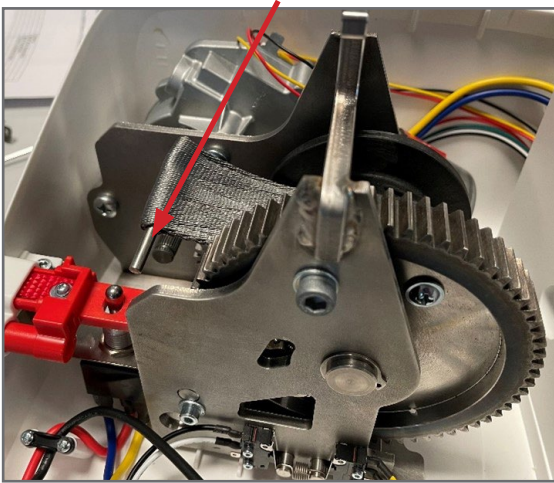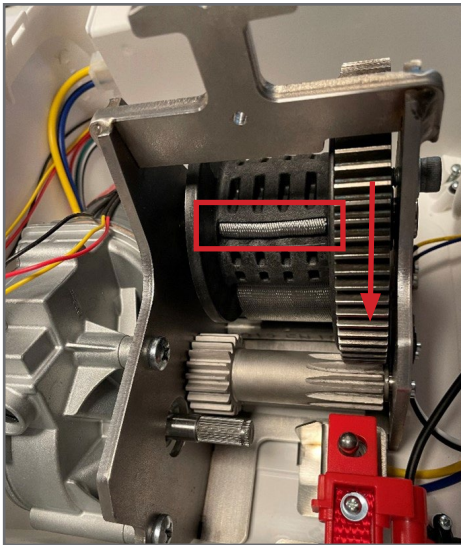
4. Put the pin into the belt loop.


5. Push back the lift belt and press/pull the lift belt so it reaches the end position in the belt roll. Roll up the lift belt in the arrow direction, see picture above.

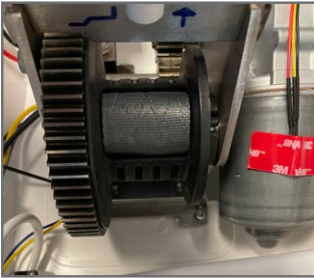ASSEMBLY INSTRUCTIONS


6. Make sure the lift belt is tight.

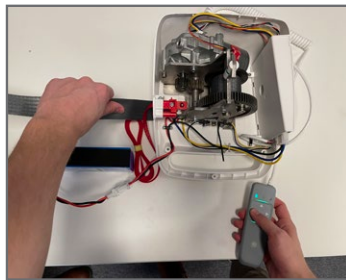7.
Deactivate the
emergency stop.

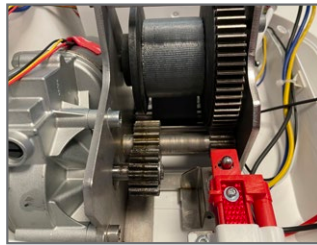8. Roll up the lift belt by use of the
hand control.

9. Keep an eye on the lift belt so it
is straight and doesn't fold.

10. Remove the Allen Key from
the sensor and mount the top
cover.

*Keep fingers
away from the
cog wheels!*

ASSEMBLY INSTRUCTIONS

Tools Required

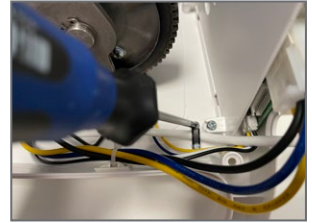
Screw driver

1.1. Remove the top cover.

1.2. Remove the hand control connector.


2. Loosen the screws holding the cable.

3. Slide the cable holder and tighten.

NOTE: check that the cable holder is properly locked in place.

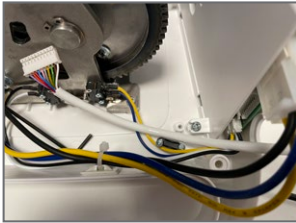
ASSEMBLY INSTRUCTIONS

1. Place the cable in the unit.

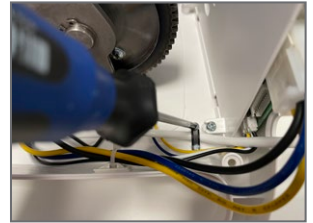
2. Connect the hand control to the PCB.

3. Tighten the cable holder.

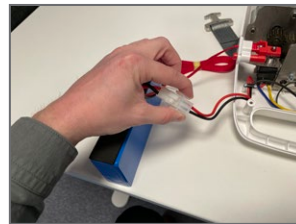4. Ensure the cable position and the rubber seal are in position.

5. Connect the battery.

6. Test the hand control. Mount the top cover.

*Keep fingers
away from the
cog wheels!*

GENERAL INFORMATION

Product name	
Article number	
Serial number	
Manufacturing date	
Location of lift	
Used firmware version	
Data log downloaded and storage location	

Inspected by	
Signature	
Date	
Approved	
Remarks	
Replaced parts	
Next service date (year and month)	

CHECKLIST

Action	Yearly service interval indication, based at expected lifetime and durability testing according to ISO 10535:2021	Passed: Yes (Y) No (N)
Mechanical components lift (cover, motor, gears, lift belt, switches etc.)		
Place the lift on a table.	Inspection for irregularities and defects that are immediately visible	Y / N
Activate the emergency stop to cut off the power.		Y / N
Remove the top cover of the lift and inspect all parts.	Inspection for irregularities and defects that are immediately visible. Check if no corrosion is visible.	Y / N
Check the cog wheels for any visible wear.	The teeth of the wheels should not show visible wear or abnormal deformation.	Y / N
Check for loose screws and tighten, if any discovered.	Lift up the product and shake the lift cassette to listen for loose parts inside.	Y / N
Inspect the hand control including the cable and the connector.	The system should be free from visible cracks or damage.	Y / N
Control the charger including the cables and the connectors.	The system should be free from visible cracks or damage.	Y / N
Assemble the cover.		Y / N
Check the UP and the DOWN function by using the buttons on top of the lift and the hand control.	Apply tension to the lift belt.	Y / N
Feed out the full length of the lift belt by pressing the DOWN-button on the hand control.	Apply tension to the lift belt.	Y / N
Check that the lift belt stops when the belt is fully pulled out.		Y / N
Check the condition of the lift belt. Check for wear and tear and replace the lift belt if it shows any sign of being worn out.	<p>The check includes visual inspection of the stitching at both sides of the lift belt.</p> <p>The lift belt holds for 2000 lifts at maximum Safe Working Load. Check the number of lifts in the data log and connect to the lifetime of the lift belt.</p> <p>The intensity and loading of the lift define the lifetime of the lift belt. The lower the lifting weight, lower frequency and symmetrical lifting reduces the service interval.</p>	Y / N

CHECKLIST

Action	Yearly service interval indication, based at expected lifetime and durability testing according to ISO 10535:2021	Passed: Yes (Y) No (N)
Mechanical components lift (cover, motor, gears, lift belt, switches etc.)		
Listen for noise or abnormal high sound.	If an abnormal sound is detected, investigate the cause to solve it.	Y / N
Check if the lift motor is working as it should.	The lift motor is designed to perform over 10 year or 11.000 lifts, whatever comes first.	Y / N
Retract the lift belt in the lift. Check that the end stop of the lift works.		Y / N
Hand control and charger		
Check that the lift turns on by pressing any key at the hand control.	The light on the hand control turns on.	Y / N
Arrows: Check the up and down function by pressing at the arrows.	The green lighted arrows are the available functions which need to be checked.	Y / N
Check the status of the battery with the battery symbol.	<p>Green light means that the battery level is full-normal.</p> <p>Green light, pulsating: it might be a good idea to charge the batteries. The symbol pulsates also during charging.</p> <p>Green light, fast pulsating: The battery level is critical low. Lift can only lower a client. The lift must be charged immediately to avoid mal function. If the symbol stays on, replace the battery.</p> <p>No light: The battery symbol disappears when in sleeping mode.</p>	Y / N
Check if warning triangle works by overloading the lift for a short moment.	Steady yellow light plus steady green light in arrow down: indicates overload, only lowering function available.	Y / N
Connect the charger to the lift and check that the green light on the charger is lit.	<p>The indicator light at the charger shows that the charger is powered.</p> <p>The indicator lights on the hand control indicates that the lift is charging.</p>	Y / N
As a last step of the yearly inspection of the lift, reset the service alarm	The lift's service indicator will lighten up with a steady yellow colour when a service needs to be performed. The service indicator is no longer lightening up after performing the service inspection.	Y / N

CHECKLIST


Action	Yearly service interval indication, based at expected lifetime and durability testing according to ISO 10535:2021	Passed: Yes (Y) No (N)
Load test		
Hang the lift in a distance strap and make sure that the lift is switched on.	The lift is attached to the rail roller in the rail.	Y / N
Active the emergency stop and check that the safety lowering function is working by pulling the red ribbon.	The lift should descend. Reset the safety lowering after checking by pressing the red button.	Y / N
Check that the emergency switch cuts the power to the lift and that the reset function is working.	The light at the hand control should not be lit when the power is cut off.	Y / N
Lower the lift/hanger bar and load it with its maximum Safe Working Load (SWL).	The Safe Working Load is stated on the product label on the lift.	Y / N
Make a full lifting cycle (at least 50cm/19,7" up and 50cm/19,7" down) to the highest level and down again.	This shall be performed in the top of the lifting range, using extensions if needed in order to prevent dangerous situations.	Y / N
Listen for noise or abnormal sound.	If an abnormal sound is detected, investigate the cause to solve it.	Y / N
Lower until the lift is unloaded. Disconnect the lift belt from the distance strap.		Y / N
Inspect the hooks of the hanger bar for damage.	No permanent deformation or cracks are accepted.	Y / N
Check that the spring on the hanger bar bends back easily.	To ensure safety, replace the springs if needed.	Y / N
Finishing lift inspection		
Check that the Product label is presence and readable and not defect in any way.	For product safety the text should be legible by the user, critical are Safe Working Load and production date to define safe use in the rail system.	Y / N
Inspection for the presence of the instructions for use and/or the user's operating instructions.	The latest version of the instruction manual can be downloaded from www.humancaregroup.com .	Y / N
Wipe off the lift with a soft cloth.	Hand a clean product over to the customer	Y / N

Reseting the service alarm

ENG

After the service is finished the service alarm should be reset.

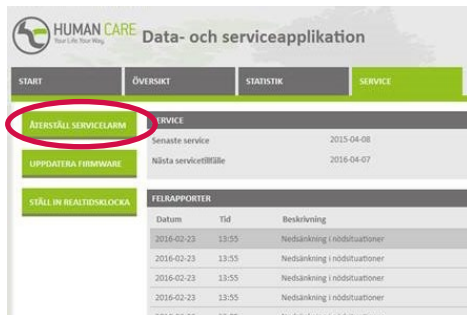- Connect the lift to a computer thru “Data and service application”. In the software, click service and log in with your login details. Click on the button “Reset service alarm”.

The screenshot shows the 'Data- och serviceapplikation' interface. The top navigation bar includes 'START', 'ÖVERSIKT', 'STATISTIK', and 'SERVICE'. Below this, there are three main sections: 'ÅTERSTÄLL SERVICEALARM' (circled in red), 'UPPDATERA FIRMVARE', and 'STÄLL IN REALTIDSLÖCKA'. The 'SERVICE' section displays a table with columns for 'Senaste service' and 'Nästa serviceöfille'. The 'FELRAPPORTER' section shows a table with columns for 'Datum', 'Tid', and 'Beskrivning'.

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